



8 CLINICAL EVALUATORS

340 TOTAL USES

98% CLINICAL RATING

Key features: High-performance curing light • Four light intensities including a 3-second **PowerCure** mode • Polyvision technology • Broad spectrum

Description

The **Bluephase® PowerCure** is an intelligent LED curing light with:

- An innovative, fully automated **Polyvision** technology, which alerts a user by vibration if the light shifts away from its initial position.
- Depending on cure time, the light will automatically adjust to avoid curing errors.
- **Polywave** LED, suitable for every light cured material.
- Four different curing and light intensity modes: High power mode, Turbo mode, 3S mode (in conjunction with the **3s PowerCure** product system) and a **Pre-Cure** mode for tack-curing materials.

Indication

- **Polywave** LED covers between 385 nm and 515 nm to cure all current dental photo-initiators and materials without limitation.



Photo courtesy of Dr. Jeff Riggs

Clinical Tips

- Incorporate all power settings, all are useful.
- Use this light with the protective shield.
- Place in charger between patient use
- Read the instructions well to learn all functions as it has many options for use.
- **3s PowerCure System** with rapid curing is extremely useful for difficult situations, especially with children, sealant placement, and gaggers.



Unique Attributes

- If the light wanders off of the targeted area while curing, the **Polyvision** will detect the motion and vibrate to alert you that you may no longer be curing the intended area.
- The **Polywave** LED covers between 385 nm and 515 nm, which covers a wide range of materials and all current dental photo-initiators.

“GOOD WEIGHT WITH A BALANCED, SLEEK, AND MODERN DESIGN.”

Evaluators' Comments

“Great curing intensity, it had different curing options and an easy swiveling curing tip.”

*“The 3-second **PowerCure** with the **PowerFill** materials are amazing. It almost feels too fast to believe.”*

“The tack cure, ‘pre-cure’ mode was spot on, making for a very consistent cleanup.”


“Cool piece of technology! We will be buying one for our practice in the future.”

“The battery charging station has a very small footprint on my counter and is easy to clean and disinfect.”


*“I already like and use the **Bluephase®** line of curing lights. The addition of the **PowerCure** settings makes this a great addition to the line.”*

“I would like a little larger tip diameter.”

Evaluation Summary:

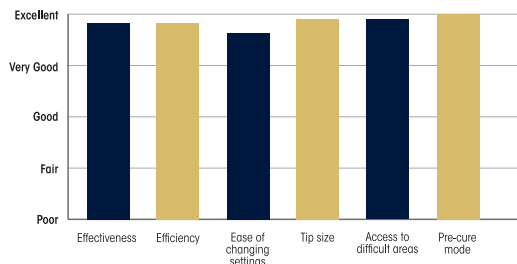

Compared to Competitive Products:

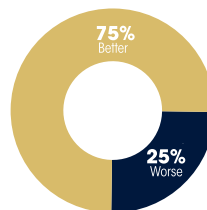

Consultants who would:

87% Recommend to a colleague

Consultants who would want to stock in office:

62% Yes, instead of current curing light

38% Yes, in addition to current curing light