Tetric® CAD

Gebruichsinstrukties

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DISCLOSURE / wear / loss of gloss.

FIVE EFFECTS / INTERACTIONS

No harmful reactions or secondary effects on patients and / or dental personnel are known.

USER / PATIENT GROUP

The product can be used by qualified dental professionals only. Suitable for all patient groups. Note that this product is not been specifically test- ed on vulnerable patient groups such as children or pregnant or lactating women.

PROPER USE

Tetric CAD must be processed using an authori- sed CAD/CAM system. For inquiries regarding the CAD/CAM system, please contact the corre- sponding cooperation partner.

INTENDED USE

Fabrication of permanent, aesthetic single-tooth restorations using a CAD/CAM system.

Stump and cavity preparation

Crowns, inlays and onlays:

- Minimal occlusal thickness 1.0 mm
- Minimal labial thickness 0.8 mm
- Minimal thickness under supporting cusps 1.0 mm

Mineral cervical thickness 0.8 mm

- Veneers:
- Minimal cervical thickness 0.3 mm

COMPOSITION

Cross-linked network of methacrylate derivatives incorporating glass fillers, amorphous silica and additives.

Komponent	wt %
Cross-linked copolymer of methacrylate derivatives	23,0–35,0
Barium-aluminum-boro-silicate glass	50,0–76,0
Amorphous silica (SiO ₂)	1,0–10,0

* Components present in very low concentrations (< 1,0 %) are not listed.

CLINICAL BENEFITS

Restoration of maxillary function

- Caries
- Endodontic treatment
- Cosmetic treatment

COMPOSITION

Cross-linked network of methacrylate derivatives incorporating glass fillers, amorphous silica and additives.

Komponent	wt %
Cross-linked copolymer of methacrylate derivatives	23,0–35,0
Barium-aluminum-boro-silicate glass	50,0–76,0
Amorphous silica (SiO ₂)	1,0–10,0

* Components present in very low concentrations (< 1,0 %) are not listed.

INDICATIONS

Permanent restoration of hard structures in the mouth (a due to:

- Caries
- Endodontic treatment
- Cosmetic treatment

KONTRAINDICATION

Contraindicated in the case of hypersensitivity to the ingredients of Tetric CAD.

SAFETY INSTRUCTIONS

Wear personal protective equipment during use.

- Use a dental dam to prevent aspiration or swallowing of dental material or other resti- dings that are detached during application. If necessary, the following maximum wall thick-

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REPORTING OBLIGATION

All serious incidents occurring in conjunction with this product must be reported immediately to the manufacturer as well as to the competent authorities. The procedure is as follows:

- Chipping / fracture / decolage
- Sensibility postoperative / irritation of the gingiva
- Secondary caries / endodontic complications
- Discoloration / wear / loss of gloss.

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Composante	peso %
Bario-aluminio-boro-silicato vetro	23,0–35,0
Barium-alumina-boro-silicate glass	50,0–76,0
Silice amorfa sublimada (SiO ₂)	1,0–10,0

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DISPOSAL

Dispose of product waste according to applica- ble legislation. Special requirements for disposa- l may apply. Dispose only after disposal of compl- y with applicable regulations.

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Tetric CAD can be modified, characterised or repaired at any time. Roughen the surface of the restoration using a diamond-coated rotary instrument. Bonding can be carried out using an adhesive suitable for this purpose (eg. Adhese Universal). Procedure according to manufacturer's instructions for use. Finally, use shades for characterisation or composite (eg. Tetric EvoFlow) according to the respective manufacturer's instructions.

SHIELD / STORAGE

Expiry date: 36 months. Use primary packaging. Storage temperature: 2–28°C / 36–82°F. Protect from heat and direct sunlight. Store in original packaging. Do not open until ready for use. Avoid extreme temperature fluctuations.

ENTSORGUNG

Dispose of product waste according to applica- ble legislation. Special requirements for disposa- l may apply. Dispose only after disposal of compl- y with applicable regulations.

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- Postoperative Sensibilität / Irritation of gingiva / Secondary caries / endodontic complications
- Discoloration / wear / loss of gloss.

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- Chipping / fracture / decolage
- Sensibility postoperative / irritation of gingiva
- Secondary caries / endodontic complications
- Discoloration / wear / loss of gloss.

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