IPS e.max® ZirCAD Prime chairside

[en] **Instructions for Use**
Zirconium oxide block (ZrO₂)

[de] **Gebrauchsinformation**
Zirkoniumoxid-Block (ZrO₂)

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Patients with permanent teeth

Intended users / Special training

- Dentists (chairside fabrication of restorations; clinical workflow)
- Dental laboratory technicians (fabrication of restorations in the dental laboratory)

No special training required.

Use

For dental use only.

Description

IPS e.max® ZirCAD Prime is a zirconium oxide block for the fabrication of fixed, full-contour restorations in anterior and posterior teeth. IPS e.max ZirCAD Prime can be processed in an authorized CAD/CAM machine.^[1]

After wet or dry processing of the block, the restorations are sintered using a sintering furnace.^[2] Once sintered, the restorations are polished using e.g. OpraGloss® Extra Oral or glazed, or characterized and glazed using IPS e.max CAD Crystall/ materials.

IPS e.max ZirCAD Prime			
Block sizes	C17, B50		
Shade range	BL1, BL3, 0, A1, A2, A3, A3.5, B1, B2, C2, D2		
Characteristics	Unit	Specifications	Typical average value
CTE (25–500 °C)	[10 ⁻⁶ • K ⁻¹]	10.5 ± 0.5	–
Flexural strength (biaxial, dentin)	[MPa]	≥ 900	1100
Fracture toughness (dentin)	[MPa • m ^{1/2}]	–	4.4 (based on ISO 14627) 5.5 (ISO 6872)
Chemical solubility	[µg/cm ²]	< 100	–
Type / Class	Type II / Class 5		

According to ISO 6872

Indications

Missing tooth structure in anterior and posterior teeth, partial edentulism in the anterior and posterior region

Types of restorations:

- Crowns
- Bridges (max. 2 connected pontics, max. 4-unit bridges)

Contraindications

- The use of the product is contraindicated if the patient is known to be allergic to any of its ingredients

Limitations of use

Do not use the product in the following cases:

- Untreated bruxism (the use of a splint is indicated after seating)
- Hybrid abutments and hybrid abutment crowns
- Temporary seating

The product must not be reused.

-  Do not reuse

In the following situations, a successful procedure cannot be ensured:

- Falling short of the required minimum wall thicknesses and minimum connector dimensions
- Milling of the blocks in a non-compatible CAD/CAM system^[3]
- Sintering in a non-compatible sintering furnace
- Mixing of IPS e.max CAD Crystall/Glaze, Shades and Stains with other dental ceramics (e.g. IPS Ivocolor Glaze, Shades and Essences)
- Use of or contact with incompatible materials

IPS e.max CAD Crystall/Glaze Spray:

- Intraoral application

Side effects

There are no known side effects to date.

Interactions

There are no known interactions to date.

Clinical benefit

- Reconstruction of chewing function
- Restoration of esthetics

Composition

Components	Percentage (%) by mass
Zirconium oxide (ZrO_2)	87.0 – 95.5
Yttrium oxide (Y_2O_3)	> 4.5 – ≤ 7.0
Hafnium oxide (HfO_2)	≤ 5.0
Aluminium oxide (Al_2O_3)	≤ 1.0
Other oxides	≤ 1.5

2 Application

The following blocks and processing techniques are available:

Processing technique			Types of restorations	
Polishing technique	Glazing technique	Staining technique	Crowns	Bridges
✓	✓	✓	C17	B50

2.1 Shade determination

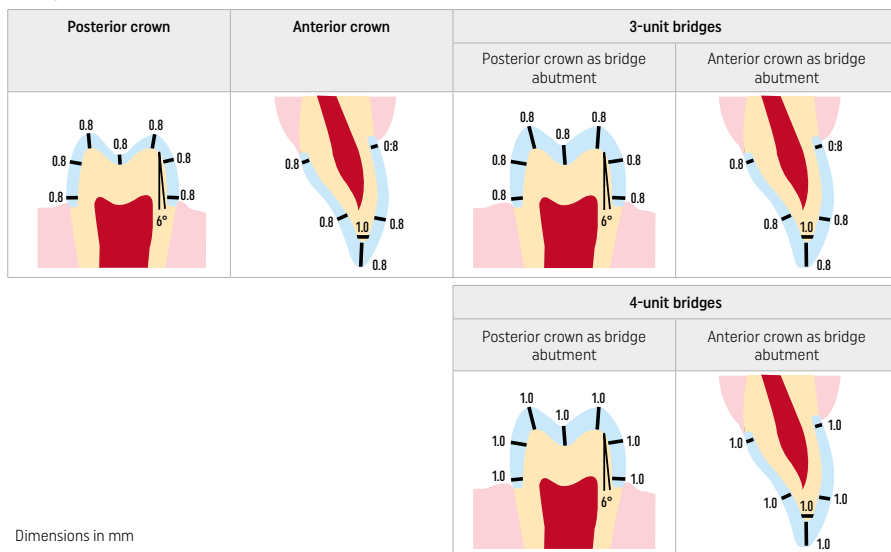
Clean the teeth prior to shade determination. Determine the shade based on the moist and unprepared tooth or the neighbouring teeth. Determine the shade of the prepared tooth based on the largest discoloured area of the preparation.

With the polishing technique, the shade effect may differ from that of the shade guide. Depending on the degree of polishing, the shade effect/intensity is increased. If necessary, it is recommended to select a block shade that is one shade brighter than the target shade.

2.2 Preparation

Prepare the tooth based on the guidelines for all-ceramic restorations and make sure to observe the minimum wall thicknesses:

- Prepare no angles or edges
- Shoulder preparation with rounded inner edges and/or chamfer preparation
- The indicated dimensions reflect the minimum wall thickness for IPS e.max ZirCAD restorations
- The radius of the preparation edges must be at least 1.0 mm (milling tool geometry) in order to ensure optimum processing in the CAD/CAM machine

Dimensions in mm

2.3 CAD/CAM processing

Make sure to observe the indicated minimum wall thicknesses and connector dimensions when designing the restoration.

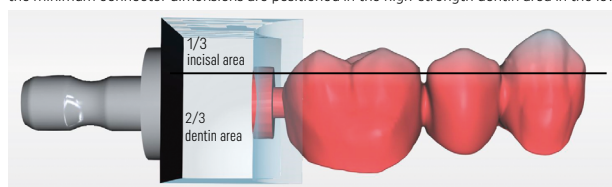
	Posterior region		Anterior region		Design type
	Minimum layer thickness in mm	Connector dimensions in mm ²	Minimum layer thickness in mm	Connector dimensions in mm ²	
Crowns	0.8	–	0.8	–	Supporting the tooth shape and/or the gingiva (Incisal, occlusal and/or basal)
3-unit bridges	0.8	9 ^[a]	0.8	9 ^[a]	
4-unit bridges	1.0	12 ^[a, b]	1.0	9 ^[a]	

^[a] Height: 4 mm, width: 3 mm

^[b] The indicated minimum connector dimensions must be positioned in the dentin area of the block.

Positioning of the designed restoration in the block (CAD):

- To ensure that restorations made of IPS e.max ZirCAD Prime show a clearly visible enamel area, they must be positioned as high up as possible on the block in the CAD software. For single tooth restorations, it is recommended to place the restoration approximately 1 mm below the top edge of the block to obtain a clearly visible incisal portion. Bridge restorations must be placed in such a way that the minimum connector dimensions are positioned in the high-strength dentin area in the lower two thirds of the block.

- Marking recess in the block: The dentin area is marked by a recess in the block. The incisal area is located on the opposite side.

Processing in the milling machine (CAM):

- **Shrinkage factor:** Since IPS e.max ZirCAD Prime shrinks by approximately 17–20% during sintering, the shrinkage factor of the respective batch, which is included in the code on the material block, must be read into the software or manually entered.
- **For the wet processing** of IPS e.max ZirCAD Prime, a dedicated milling fluid container should be used to avoid cross-contamination with other materials (e.g. milling dust). Do not exceed the maximum amount of milling fluid for wet processing. Contamination may result in discolouration of the restorations during sintering.
- **For dry processing** make sure that any residue is removed from the chamber of the CAM unit and that the chamber is dry.

2.4 Finishing

When separating the restorations from the block, make sure they are not contaminated with any grease or smudgy film since this may negatively influence the shade result.

General information

- Non-dense-sintered zirconium oxide restorations are susceptible to damage and fractures. This fact has to be kept in mind during the entire working procedure.
- If possible, any necessary post-processing procedure should be carried out while the restoration is still in its non-dense-sintered state (observe the recommendations regarding grinding instruments^[3]).
- In the non-dense-sintered state, the contact with unsuitable liquids or liquids not approved for zirconium oxide (e.g. contaminated water and/or coolant liquid) and/or contact media (e.g. occlusal spray) must be prevented.
- Only use suitable grinding instruments, low speed and light pressure to prevent delamination and chipping at the margins in particular (observe the recommendations regarding grinding instruments^[3]).
- The non-dense-sintered restoration must not be cleaned in an ultrasonic bath or with the steam jet.

Dry-milled restorations

- Carefully separate the restoration from the holder using a separating disc or suitable grinding instruments^[3]. Smooth out the attachment points of the holding bars with suitable grinding instruments^[3].
- After finishing, thoroughly remove any zirconium oxide dust either with a soft brush or by blasting the restoration with oil-free compressed air.

Wet-milled restorations

- Carefully separate the restoration from the holder using a separating disc or suitable grinding instruments^[3]. Smooth out the attachment points of the holding bars with suitable grinding instruments^[3].
- After finishing, thoroughly remove any adhering zirconium oxide dust from the restoration. Wet-milled restorations may be rinsed with slowly running water.
- In order to prevent damage to the restoration during sintering, the IPS e.max ZirCAD restoration must be completely dry. Moist restorations must not be sintered.

General notes on finishing

- Coarse tungsten carbide burs and/or grinding instruments^[3] with large diameters are only suitable to a certain extent, since they may cause vibrations during finishing, which may result in chipping, among other things. Therefore, only fine tungsten carbide burs and/or grinding instruments^[3] should be used.

- Do not "post-separate" the connector area of bridge constructions using a separating disc as this leads to predetermined breaking points. Make sure that the minimum wall thicknesses and connector dimensions are maintained during finishing.
- Make sure to remove any adhering zirconium oxide dust before sintering. Adhering zirconium oxide particle may be sintered to the restoration and result in inaccuracies of fit.
- The restoration must not be blasted with Al_2O_3 or glass polishing beads.

2.5 Sintering

Sintering^[2] is a thermal process during which IPS e.max ZirCAD Prime obtains its final physical and mechanical properties, such as its high strength and translucency.

General notes

- Sintering temperatures that are too low or too high and/or sintering times that are too short or to too long will have a negative effect on the above mentioned final properties.
- Basically, sintering beads are not recommended for the Programat CS6 sintering furnace being used.
- The restorations must not come into contact with each other during sintering.
- The correct program selection must be observed.
- Always keep the sintering accessories clean and free of dust so that the sintered restorations are not contaminated.
- For correct placement of the restorations in the sintering furnace, please observe the notes of the furnace being used.
-  The respective Operating Instructions of the sintering furnace^[2] in use must be observed.

Process after sintering

- Always allow the restorations to cool to room temperature before proceeding.
- Do not touch the hot objects with metal tongs.
- Do not blast or quench the objects.

2.6 Finishing process

- Processing of sintered restorations should be kept to a minimum.
- The restoration should only be mechanically processed if it is absolutely necessary.
- Use only low pressure for reworking.
- If diamond grinding instruments are used for adjustments, the restoration must be cooled with water. As an alternative, other grinding instruments may be used in conjunction with a dental-lab handpiece.
- Please observe the manufacturer's grinding tool recommendations.
- Check marginal areas and slightly rework, if necessary.
- When reworking the restorations, avoid designing sharp edges, aim for rounded ones.
- Do not use a separating disc to separate interdental areas.
- Make sure the minimum wall thicknesses and connector dimensions are maintained during reworking.
- Use only immaculate grinding instruments.
- Check the restoration for defects and tears before and after further processing.
- Zirconium oxide dust sintered to the restoration must be removed before glazing.
- Then clean the restoration with running water or the steam jet and dry.

2.7 Completion

Before proceeding, pre-finish and finish at least the occlusal contact surfaces. Additionally smooth the basal side of the framework connectors in bridges using rubber polishers.

A first pre-polishing is possible before sintering to ease the effort of polishing for users. For this purpose, use only OptraGloss Extra Oral polishers in the shape "Spiral Wheel". Make sure to use only light contact pressure and a speed of max. 5000 rpm (dark blue) and 10,000 rpm (light blue). Pre-polishing for an excessive amount of time can lead to a deterioration of the polishing result. Blue residue from the polisher should be avoided as much as possible.

Well-polished surfaces help to protect the opposing teeth from undesired wear. When doing this, make sure that the surface of the ground occlusal surfaces are carefully polished after functional adjustment by grinding (observe the recommendations regarding grinding instruments^[3]). Various approaches are available to finish restorations.

Polishing

When final polishing, polish all areas of the outer surfaces. This minimizes plaque build-up and achieves an esthetically pleasing result. Polishers that are applied extraorally are suitable for this purpose, e.g. OptraGloss® Extra Oral.

- 1) Pre-finishing: Use OptraGloss Extra Oral (diamond prepolymer/dark blue) dry at low speed to create a smooth overall surface.
- 2) Finishing: Finish functional areas using OptraGloss Extra Oral (diamond high-gloss polisher/light blue).
- 3) Optional: High-gloss polishing using OptraGloss polishing paste

Glazing and staining technique

Complete the restoration either by glazing only or by staining and glazing in one step.

- Glazing technique

a) Glazing using IPS e.max CAD Crystall./Glaze Spray

Position the restoration as described under "Fixing the restoration on an IPS e.max CAD Crystallization Pin". Hold the restoration by the IPS e.max CAD Crystallization Pin. Shake the IPS e.max CAD Crystall./Glaze Spray vigorously immediately before use until the mixing ball in the container is moving freely (approximately 20 seconds). If the spray is not sufficiently shaken, mainly the propellant is discharged with a spraying burst. As a result, the restoration is insufficiently coated with glazing powder. Observe a distance of 10 cm between the nozzle and the surface to be sprayed. Hold the spray can as upright as possible during spraying. Spray the restoration from all sides with short bursts while simultaneously rotating the restoration so that an even, covering layer is created. Shake the can again between individual bursts of spray. Subsequently, spray the restoration a second time from all sides with short bursts while simultaneously rotating the restoration so that an even, covering layer is created. Shake the can again between individual bursts of spray. Wait briefly until the glaze layer is dry and has assumed a whitish colour. Areas that do not show an even layer have to be sprayed again. Fire the restoration using the stipulated firing parameters. Ensure that the maximum loading capacity of the sintering furnace is not exceeded. Observe the directions under "How to proceed after firing". For adjustments, please observe the information under "Corrective firing".

b) **Glazing** using IPS e.max CAD Crystall./Glaze Paste/Fluo

Position the restoration as described under "Fixing the restoration on an IPS e.max CAD Crystallization Pin". Subsequently, apply a covering layer of IPS e.max CAD Crystall./Glaze Paste/Fluo evenly on the entire outer surfaces of the restoration using a brush. If the ready-to-use glaze needs to be slightly diluted, it may be mixed with a small amount of IPS e.max CAD Crystall./Glaze Liquid. Do not apply the glaze too thickly. Avoid "pooling", especially on the occlusal surface. Too thin a glaze layer may lead to an unsatisfactory gloss. If the IPS e.max CAD Crystall./Glaze Paste/Fluo accidentally reaches the inner aspects of the restoration, remove it with a dry short-hair brush before firing. Fire the restoration using the stipulated firing parameters. Ensure that the maximum loading capacity of the sintering furnace is not exceeded. Observe the directions under "How to proceed after firing". For adjustments, please observe the information under "Corrective firing".

– **Staining technique**

c) **Staining and glazing** using IPS e.max CAD Crystall./Glaze Spray

Staining and glazing with IPS e.max CAD Crystall./Glaze Spray takes place in one firing cycle. Position the restoration as described under "Fixing the restoration on an IPS e.max CAD Crystallization Pin". Extrude the ready-to-use shades or stains from the syringe and mix them thoroughly. The shades and stains may be slightly thinned using IPS e.max CAD Crystall./Glaze Liquid. However, the consistency should still remain pasty. Apply the mixed shades and stains directly to the restoration using a fine brush. Spray the restoration with IPS e.max CAD Crystall./Glaze Spray from all sides to coat it with an even layer. Proceed according to the instructions under a). Fire the restoration using the stipulated firing parameters. Ensure that the maximum loading capacity of the sintering furnace is not exceeded. Observe the directions under "How to proceed after firing". For adjustments, please observe the information under "Corrective firing".

d) **Staining and glazing** using IPS e.max CAD Crystall./Glaze Paste/Fluo

Staining and glazing of IPS e.max CAD Crystall./Glaze Paste/Fluo takes place in one firing cycle. Position the restoration as described under "Fixing the restoration on an IPS e.max CAD Crystallization Pin". Subsequently, apply IPS e.max CAD Crystall./Glaze Paste/Fluo evenly on the outer surfaces of the restoration using a brush. If the ready-to-use glaze needs to be slightly diluted, it may be mixed with a small amount of IPS e.max CAD Crystall./Glaze Liquid. Do not apply the glaze too thickly. Avoid "pooling", especially on the occlusal surface. Too thin a glaze layer may lead to an unsatisfactory gloss. If characterizations are desired, the restoration can be individualized using IPS e.max CAD Crystall./Shades and/or IPS e.max CAD Crystall./Stains. Extrude the ready-to-use shades and stains from the syringe and mix them thoroughly. The shades and stains may be slightly thinned using IPS e.max CAD Crystall./Glaze Liquid. However, the consistency should still remain pasty. Apply the mixed shades and stains directly to the unfired glaze layer using a fine brush. Place the restoration in the centre of the IPS e.max CAD Crystallization Tray. A maximum of 6 units can be positioned on the firing tray for Glaze firing. Position the IPS e.max CAD Crystallization Tray on the sintering tray and fire using the stipulated firing parameters. Observe the directions under "How to proceed after firing". For adjustments, please observe the information under "Corrective firing".

– **Corrective firing**

If additional characterizations or adjustments are required, a corrective firing using IPS e.max CAD Crystall./Shades, Stains and Glaze can be conducted. Also use the IPS e.max CAD Crystallization Tray on the sintering tray for the corrective firing cycle. Use the IPS e.max CAD Crystall./Add-On including the respective mixing liquid to carry out minor shape adjustments (e.g. proximal contact points). Apply the mixed add-on material directly on the unfired Glaze Paste and/or Shades and Stains in the areas to be adjusted and fire. If the Glaze Spray is used, apply the Shades and Stains first. Subsequently, supplement the missing areas using Add-On. Apply the Glaze Spray immediately after the application of the Add-On and fire. After having applied the add-on material, conduct corrective firing.

Fixing the restoration on an IPS e.max CAD Crystallization Pin

1. Select the largest possible IPS e.max CAD Crystallization Pin (S, M, L) that best "fills" the inside of the restoration, but does not come into contact with the surrounding crown walls.
2. Subsequently, fill the inside of the restoration with IPS Object Fix Putty or Flow up to the restoration margin. Immediately re-seal the IPS Object Fix Putty/Flow syringe after extruding the material. Once removed from the aluminium bag, the syringe is ideally stored in a resealable plastic bag or a container with a humid atmosphere.
3. Press the selected IPS e.max CAD Crystallization Pin deeply into the IPS Object Fix Putty or Flow material so that it is adequately secured.
4. Prevent any IPS Object Fix residue from adhering to the outer surface of the restoration. Clean off any possible contamination with a brush dampened with water and then dry.

How to proceed after firing

Remove the restoration from the furnace once the firing cycle is complete (wait for the acoustic signal of the furnace) and allow it to cool to room temperature in a place protected from draft. Hot objects must not be touched with metal tongs. Remove the restoration from the hardened IPS Object Fix Putty / Flow. Remove any residue with ultrasound in a water bath or with the steam jet. If the restoration needs to be adjusted by grinding³, make sure that no overheating of the ceramic occurs.

Shade combination table

Customized characterizations and shade adjustments are applied using IPS e.max CAD Crystall./Shades and Stains.

A–D shade	BL1	BL2	BL3	BL4	0	A1	A2	A3	A3.5	A4	B1	B2	B3	B4	C1	C2	C3	C4	D2	D3	D4
IPS e.max CAD Crystall./ Shade	0				1						2				3				4		
IPS e.max CAD Crystall./ Shade Incisal	I1								I2		I1				I2						
IPS e.max CAD Crystall./ Stains	white				cream			sunset			copper			olive		khaki			mahogany		

Firing parameters

IPS e.max ZirCAD Prime

Corrective/Stains/Glaze firing with IPS e.max CAD Crystall./ materials

Furnace Programat		Stand-by temperature B [°C/°F]	Closing time S [min]	Heating rate t ↗ [°C/°F/min]	Firing temperature T1 [°C/°F]	Holding time H1 [min]	Heating rate t ↗ [°C/°F/min]	Firing temperature T2 [°C/°F]	Holding time H2 [min]	Vacuum 1 11 12 [°C/°F]	Vacuum 2 21 22 [°C/°F]	Long-term cooling L [°C/°F]	Cooling rate tl [°C/°F/min]
CS6 ^(a)		–	Glaze ^(c)	150/270	700/1292	00:00	75/135 30/54 ^(d)	830/1526 890/1634 ^(d)	00:00 00:20 ^(d)	50/122	815/1499	850/1562	-70/-126
CS4 ^(b)		180/356 ^(a) 230/446 ^(a)	00:00 03:00 ^(c) 03:00 ^(c)	60/108	400/752	00:00	95/171 45/81 ^(d)	840/1544 900/1652 ^(d)	0 3 ^(d)	–	–	890/1634	-70/-126
CS3		403/757	06:00	90/162	830/1526	00:10	30/54	870/1598	03:00	450/842 830/1526	830/1526 869/1596	0	0
CS2		403/757	06:00	90/162	830/1526	00:10	30/54	870/1598	03:00	450/842 830/1526	830/1526 869/1596	0	0
CS		403/757	06:00	90/162	820/1508	00:10	30/54	860/1580	03:00	450/842 820/1508	820/1508 859/1578	0	0

IPS e.max ZirCAD Prime

Corrective/Stains/Glaze firing with IPS e.max CAD Crystall./ Add-On

Furnace Programat		Stand-by temperature B [°C/°F]	Closing time S [min]	Heating rate t ↗ [°C/°F/min]	Firing temperature T1 [°C/°F]	Holding time H1 [min]	Heating rate t ↗ [°C/°F/min]	Firing temperature T2 [°C/°F]	Holding time H2 [min]	Vacuum 1 11 12 [°C/°F]	Vacuum 2 21 22 [°C/°F]	Long-term cooling L [°C/°F]	Cooling rate tl [°C/°F/min]
CS6		–	Glaze ^(c)	150/270	700/1292	00:00	75/135 30/54 ^(d)	830/1526 890/1634 ^(d)	00:00 00:20 ^(d)	50/122	815/1499	850/1562	-70/-126
CS4	–	–	–	–	–	–	–	–	–	–	–	–	–
CS3		403/757	06:00	90/162	830/1526	00:10	30/54	870/1598	03:00	450/842 830/1526	830/1526 869/1596	0	0
CS2		403/757	06:00	90/162	830/1526	00:10	30/54	870/1598	03:00	450/842 830/1526	830/1526 869/1596	0	0
CS		403/757	06:00	90/162	820/1508	00:10	30/54	860/1580	03:00	450/842 820/1508	820/1508 859/1578	0	0

^(a) IPS CAD Speed Tray, max. 3 units or one bridge

^(b) Place IPS e.max CAD Crystallization Tray on the sintering tray, max. 6 units

^(c) Values for pre-drying

^(d) Third heating stage

2.8 Seating

Prior to seating, condition the IPS e.max ZirCAD Prime restoration by sandblasting the inner aspects of the crown with Al₂O₃, 25–70 µm, 1 bar (14.5 psi) or Al₂O₃, 70–110 µm, 1.5 bar (21.75 psi).

	Adhesive cementation e.g. Multilink® Automix	Self-adhesive cementation e.g. SpeedCEM® Plus	Conventional cementation e.g. ZirCAD® Cement
Preparation requirements	Non-retentive preparation	Retentive preparation	Retentive preparation
Sandblasting	Al ₂ O ₃ , 25–70 µm, 1 bar (14.5 psi) or Al ₂ O ₃ , 70–110 µm, 1.5 bar (21.75 psi)		
Cleaning after try-in	e.g. Ivoclean		
Conditioning	60 s with e.g. Monobond® Plus	–	
Crowns	✓		
Bridges	✓		

3 Safety information

- In the case of serious incidents related to the product, please contact Ivoclar Vivadent AG, Bendererstrasse 2, 9494 Schaan/Liechtenstein, website: www.ivoclar.com, and your responsible competent authority.
- The current Instructions for Use are available on the website (www.ivoclar.com/elifu).
- Explanation of the symbols: www.ivoclar.com/elifu
- The Summary of Safety and Clinical Performance (SSCP) can be retrieved from the European Database on Medical Devices (EUDAMED) at <https://ec.europa.eu/tools/eudamed>.
Basic UDI-DI: 76152082ACERA006F2

Warnings

- Observe the Safety Data Sheet (SDS) (available at www.ivoclar.com).
- The processing of IPS e.max ZirCAD Prime blocks produces dust which may irritate the skin and eyes and which may result in lung damage. Make sure that the suction equipment at your milling machine and at your workplace works flawlessly. Do not inhale grinding dust during finishing and wear a dust mask (particle class FFP2) as well as protective goggles and gloves.
- IPS e.max CAD Crystall./Glaze Spray: Do not inhale the spray mist. The container is pressurized. Protect from direct sunlight and temperatures over 50 °C/122 °F. Do not forcibly open or burn the can, even after it has been completely emptied.

Disposal information

Remaining stocks or removed restorations must be disposed of according to the corresponding national legal requirements.

Residual risks

Users should be aware that any dental intervention in the oral cavity involves certain risks. The following clinical residual risks are known:

- Chipping / fracture / decementation of the restorative material may lead to accidental swallowing or inhalation and dental re-treatment.
- Cement excess may lead to irritation of the soft tissue / gingiva. Progressive inflammation may lead to bone resorption or periodontitis.

4 Shelf life and storage

- In the original packaging
- In a dry place
- Do not expose to physical impact or vibrations.

5 Additional information

Keep material out of the reach of children!

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of use. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions.

^[1] The complete list of compatible CAD/CAM systems is available at www.ivoclar.com.

^[2] e.g. Programat CS6, Programat CS4, CEREC SpeedFire. CEREC SpeedFire is not a registered trademark of Ivoclar Vivadent AG.

^[3] Ivoclar Vivadent Flow Chart "Recommended grinding instruments for extraoral and intraoral use"

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