OptraDam® Plus

Brugsanvisning

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Bruksanvisning

Productinformatie

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Instructions for Use

Description

OptraDam® Plus is an anatomically shaped rubber dam for the absolute isolation of the working field. It provides retraction of lips and cheeks during dental treatment and offers the clinician increased visibility and accessibility. In addition, the device facilitates the evacuation of water spray. OptraDam Plus features high flexibility and elasticity in all directions of movement, provides patient comfort and assists in keeping the patient's mouth open. OptraDam Plus is made of latex and powdered with corn starch. OptraDam Plus is available in regular and small sizes.

Indication

Suitable for use in therapeutic dental procedures for which an absolutely dry and isolated working field is required (e.g. for adhesive cementation, filling therapy and root canal treatments).

Contraindication

The use of OptraDam Plus is contraindicated for patients with latex allergy.

Description of the procedure

When OptraDam Plus is used, the small intraoral ring of the device is positioned in the area of the gingivobuccal fold, while the outer ring remains outside the mouth. The elastic component between the two rings thus embraces the lips of the patient and provides retraction due to the restoring force of the rings (see Figure 1).

Fig 1: OptraDam Plus in place

1. Select the appropriate sized OptraDam Plus. The size (R for regular or S for small) is indicated on the packaging and on the back of the latex bag.
2. The rubber dam punch is used to make the appropriate holes, depending on the teeth to be exposed. Check the quality of your rubber dam punch regularly. Blunt punches perforate the latex only incompletely, which results in the latex material tearing and thus in insufficient isolation of the teeth.
3. OptraDam Plus may be inserted from both the 12 o'clock position with the patient lying down, or from the 8 o'clock position with the patient sitting upright (or, the 4 o'clock position if the clinician is left-handed).
4. To ease placement, the operator should grasp the intraoral ring by reaching through the outer ring. The intraoral ring should be held between the thumb and middle finger and pressed together slightly (see Figure 2).

When grasping the intraoral ring, make sure that it is not completely pulled out of the latex bag.

Fig 2: Correct hold on the inner ring with thumb and middle finger. The ring is slightly compressed in the process.

5. The slightly compressed intraoral ring is then inserted into the buccal corridor on one side, so that the elastic component embraces the corner of the mouth and the outer ring is still located extraorally (see Figure 2).
6. As soon as OptraDam Plus is securely in place on one side, the intraoral ring is inserted in the same manner on the other side by slightly bending it (see Figure 2).

Fig 3: After having positioned the inner ring behind the corners of the mouth, it is pushed behind the lower and upper lip.

7. Subsequently, the inner ring is placed behind the lower and upper lips and thus the OptraDam Plus assumes its final, stable position (see Figure 3). The intraoral ring can be inserted with more ease if the patient's mouth is relaxed. If, in individual cases, the intraoral ring should show a tendency to slip out of the mouth when the mouth is closed completely, positioning of the intraoral ring deeper inside the vestibule is usually sufficient to solve the problem. Sometimes, the use of the other size may also be a solution.
 8. The dam is secured around the teeth by fitting the septum of the dam interproximally and in the sulcus using dental floss.
 9. The securely positioned OptraDam Plus improves the overall view and accessibility of the working field. The saliva ejector is introduced below the integrated frame and positioned in the corner of the mouth. Lateral movements of the mandible and opening of the mouth to various degrees may provide additional space and access to the treatment area. If the interproximal spaces are large and retention of the rubber dam is insufficient, a ligature of dental floss or Wedjets®* or a rubber dam clamp may be used additionally to hold the latex bag in place.
- * Wedjets® is a registered trademark of Coltène/Whaledent
10. Before the dam is removed, each septum is carefully cut. Using a paper towel, grasp the outer ring in the lower area and lift it slightly so that the intraoral ring is loosened from the lower gingivobuccal fold. The

intraoral ring can then be easily removed from the upper gingivobuccal fold and hygienically disposed of with the paper towel.

Note

- OptraDam Plus is for single patient use only.
- In patients with impeded nose respiration or if suction is desired, a hole may be cut into the latex bag in the palatal region using scissors.
- OptraDam Plus contains natural rubber latex which may cause allergic reactions.
- In general, it is possible to take X-ray images with OptraDam Plus in place if sufficiently rubber material is available intraorally – if necessary, pull in more of the latex material.
- If an anaesthetic has to be administered again, OptraDam Plus can be easily removed.

Storage

- Storage temperature: 2–28°C / 36–82°F
- Keep out of direct sunlight

Keep out of the reach of children!

For use in dentistry only!

Date of preparation: 03/2009

Revision 0

Manufacturer

Ivoclar Vivadent AG, FL-9494 Schaan/Liechtenstein

The material has been developed solely for use in dentistry. Processing should be carried out strictly according to the Instructions for Use. Liability cannot be accepted for damages resulting from failure to observe the Instructions or the stipulated area of application. The user is responsible for testing the products for their suitability and use for any purpose not explicitly stated in the Instructions. Descriptions and data constitute no warranty of attributes and are not binding.

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Ivoclar Vivadent UK Limited
Ivoclar Vivadent, Inc.