Description

Tetric EvoCeram® Bulk Fill is a state-of-the-art, light-curing, radiopaque, nano-hybrid composite for direct restorations in posterior teeth. Tetric EvoCeram Bulk Fill cures with light in the wavelength range of 400–500 nm (blue light) and can be applied in layers of up to 4 mm.

Shades

Tetric EvoCeram Bulk Fill is available in the following 3 shades:

- Universal shades: IVA, IVB, IVW

Composition

The monomer matrix is composed of dimethacrylates (20–21% weight). The fillers contain barium glass, ytterbium trifluoride, mixed oxide and prepolymer (79–81% weight). Additional contents: additives, catalysts, stabilizers and pigments (<1.0% weight). The total content of inorganic fillers is 76–77% weight or 53–54% volume. The particle size of the inorganic fillers is between 40 nm and 3,000 nm with a mean particle size of 550 nm.

Indication

- Restorations of deciduous teeth
- Restorations in the posterior region (Classes I and II, including the replacement of individual cusps)
- Class V restorations (cervical caries, root erosion, wedge-shaped defects)
- Extended fissure sealing in molars and premolars

Contraindication

Placement of Tetric EvoCeram Bulk Fill restorations is contra-indicated:

- if a dry working field cannot be established, or if the stipulated technique cannot be applied;
- if a patient is known to be allergic to any of the ingredients in Tetric EvoCeram Bulk Fill.

Side effects

In individual cases, components of Tetric EvoCeram Bulk Fill may lead to sensitization. Tetric EvoCeram Bulk Fill should not be used in such cases. To avoid possible irritation of the pulp, areas close to the pulp should be protected with a suitable pulp/dentin protector (selectively apply a calcium hydroxide-based preparation in areas close to the pulp and cover with suitable cavity liner).

Interactions

Materials containing phenolic substances, e.g. eugenol/clove oil, inhibit the polymerization of methacrylate-based materials. Consequently, application of such materials together with Tetric EvoCeram Bulk Fill must be avoided. Discolouration may occur in combination with cationic mouthwashes, plaque disclosing agents and chlorhexidine.

Application

1. Shade determination

Clean the teeth prior to shade determination. The shade is selected with the tooth still moist.

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2. Isolation

Appropriate isolation, best with a rubber dam (e.g. OptraDam® Plus), is required.

3. Cavity preparation

Cavity preparation is carried out according to the requirements of the adhesive technique, i.e. protecting the tooth structure. Do not prepare sharp, internal edges and angles or additional undercuts in caries-free areas. The dimensions of the cavity are generally determined by the extent of the caries or the size of the old restoration. Bevel enamel edges in the anterior region. For the posterior region, only the sharp enamel edges should be rounded (finishing diamonds, 25–40 µm). Caries-free cervical defects are not prepared, only cleaned with pumice or other suitable cleaning pastes with the help of rubber cups or rotary brushes. Subsequently, remove all residue in the cavity with water spray and dry with water- and oil-free air.

4. Pulp protection / Base

Do not apply a base material when using an enamel/dentin bonding agent. Only cover very deep areas close to the pulp with a calcium hydroxide material (e.g. ApexCal®) and subsequently use a pressure-resistant cement (e.g. a glass ionomer cement, such as Vivaglass® Liner). Do not cover other cavity walls, since they can be used to support the bond with an enamel/dentin adhesive.

5. Matrix / Interdental wedge

Use a wrap around matrix for cavities affecting the proximal area or a sectional matrix and wedge it.

6. Conditioning / Application of the bonding agent

Condition and apply the bonding agent according to the Instructions for Use of the product in use. We recommend using Syntac® (with phosphoric acid etching) or Excite® F (with phosphoric acid etching) or the self-etching adhesive AdheSE®.

7. Application of Tetric EvoCeram Bulk Fill

- For an optimum result, apply Tetric EvoCeram Bulk Fill in layers of max. 4 mm and adapt with a suitable instrument (e.g. OptraSculpt). Use a special contact point instrument (e.g. the bifurcated OptraContact) for large cavities where the contacts are difficult to create.
- Sufficient exposure to the curing light prevents incomplete polymerization. For the recommendations regarding exposure time and light intensity see Table 1.
- When using a metal matrix, additionally polymerize the composite material from the buccal or lingual/palatal aspect after removing the matrix, if no Bluephase polymerization light is used or the light probe cannot be ideally positioned, e.g. distant to composite or diverging scattering angle.
- In many cases, a flowable composite is used today as an initial layer to create an even cavity floor and to facilitate the adaptation of the subsequently used restorative material. A flowable composite (e.g. Tetric EvoFlow®) can be used as a thin initial layer. This optional layer has to be cured separately (please refer to the respective Instructions for Use).

8. Finishing / Checking the occlusion / Polishing

Remove excess material with suitable finishers or fine diamonds after polymerization. Remove proximal excess with diamond or tungsten carbide finishers, finishing strips, or flexible finishing discs. Check the occlusion and articulation and apply appropriate grinding corrections to prevent premature contacts or undesired articulation paths on the surface of the restorations. Use silicone polishers (e.g. OptraPol® Next Generation) as well as polishing discs and polishing strips to polish the restorations to a durable high gloss.

Additional information

1. Tetric EvoCeram Bulk Fill can be used in combination with Tetric EvoCeram and Tetric EvoFlow.
2. In the case of repair, additional Tetric EvoCeram Bulk Fill can be directly applied to polymerized material. If the Tetric EvoCeram Bulk Fill restoration has already been polished, it must first be roughened and wetted with Heliobond before fresh Tetric EvoCeram Bulk Fill is applied.
3. Tetric EvoCeram Bulk Fill should have ambient temperature when applied. Cool temperatures render the material difficult to extrude.
4.  For single use only. If Tetric EvoCeram Bulk Fill is applied from the Cavifil directly in the mouth of the patient, the Cavifil must not be used for more than one patient due to hygienic reasons (prevention of cross-contamination between patients).
5. Syringes or Cavifils should not be disinfected with oxidizing disinfection agents.
6. The recommended increment thickness is based on hardness profile measurements.

Warning

Unpolymerized Tetric EvoCeram Bulk Fill should not come in contact with skin, mucous membrane, or eyes. Unpolymerized Tetric EvoCeram Bulk Fill may have a slight irritating effect and may lead to a sensitization against methacrylates. Commercial medical gloves do not provide protection against the sensitizing effect of methacrylates.

Storage

- Storage temperature 2–28 °C / 36–82 °F.
- Close syringes / Cavifils immediately after usage. Exposure to light causes premature polymerization.
- Do not use Tetric EvoCeram Bulk Fill after the indicated date of expiration.
- Shelf life: see information on Cavifils, syringes and packages.

Keep material out of children's reach.

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Table 1

Unit Program	Bluephase C8	Bluephase Style	Bluephase	Bluephase 20i
Turbo	–	–	–	5 s
High Power	15 s	10 s	10 s	10 s
Soft Start	20 s	–	15 s	15 s

Light Intensity	Exposure Time
≥ 500 mW/cm ²	20 s
≥ 1000 mW/cm ²	10 s

Ivoclar Vivadent AG

Bendererstrasse 2 | 9494 Schaan | Liechtenstein

Tel. +423 235 35 35 | Fax +423 235 33 60 | www.ivoclarvivadent.com

Ivoclar Vivadent Pty. Ltd.

1 – 5 Overseas Drive | P.O. Box 367 | Noble Park, Vic. 3174 | Australia

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Tel. +1 905 670 8499 | Fax +1 905 670 3102 | www.ivoclarvivadent.us

Ivoclar Vivadent (Shanghai) Trading Co., Ltd.

2/F Building 1, 881 Wuding Road, Jing An District | 200040 Shanghai | China

Tel. +86 21 6032 1657 | Fax +86 21 6176 0968 | www.ivoclarvivadent.com

Ivoclar Vivadent Marketing Ltd.

Calle 134 No. 7-B-83, Of. 520 | Bogotá | Colombia

Tel. +57 1 627 33 99 | Fax +57 1 633 16 63 | www.ivoclarvivadent.com

Ivoclar Vivadent SAS

B.P. 118 | F-74410 Saint-Jorioz | France

Tel. +33 450 88 64 00 | Fax +33 450 68 91 52 | www.ivoclarvivadent.fr

Ivoclar Vivadent GmbH

Dr. Adolf-Schneider-Str. 2 | D-73479 Ellwangen, Jagst | Germany

Tel. +49 (0) 79 61 / 8 89-0 | Fax +49 (0) 79 61 / 63 26 | www.ivoclarvivadent.de

Wieland Dental + Technik GmbH & Co. KG

Schwenninger Strasse 13 | D-75179 Pforzheim | Germany

Tel: +49 (0) 72 31 / 37 05 - 0 | Fax: +49 (0) 72 31 / 35 79 59 |

www.wieland-dental.com

Ivoclar Vivadent Marketing (India) Pvt. Ltd.

503/504 Raheja Plaza | 15 B Shah Industrial Estate |

Veera Desai Road, Andheri (West) | Mumbai, 400 053 | India

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Tel. +7 499 418-03-00 | Fax +7 499 418-03-10 | www.ivoclarvivadent.ru

Ivoclar Vivadent Marketing Ltd.

Qlaya Main St. | Siricon Building No.14, 2nd Floor | Office No. 204 |

P.O. Box 300146 | Riyadh 11372 | Saudi Arabia

Tel. +966 1 293 83 45 | Fax +966 1 293 83 44 | www.ivoclarvivadent.com

Ivoclar Vivadent Pte. Ltd.

171 Chin Swee Road | #02-01 San Centre | Singapore 169877

Tel. +65 6535 6775 | Fax +65 6535 4991 | www.ivoclarvivadent.com

Ivoclar Vivadent S.L.U.

c/ Emilio Muñoz Nº 15 | Entrada c/ Albarracin | E-28037 Madrid | Spain

Tel. + 34 91 375 78 20 | Fax + 34 91 375 78 38 | www.ivoclarvivadent.es

Ivoclar Vivadent AB

Dalvägen 14 | S-169 56 Solna | Sweden

Tel. +46 (0) 8 514 93 930 | Fax +46 (0) 8 514 93 940 | www.ivoclarvivadent.se

Ivoclar Vivadent Liaison Office

: Tesvikiye Mahallesi | Sakayik Sokak | Nisantasi' Plaza No:38/2 |

Kat:5 Daire:24 | 34021 Sisli – Istanbul | Turkey

Tel. +90 212 343 08 02 | Fax +90 212 343 08 42 | www.ivoclarvivadent.com

Ivoclar Vivadent Limited

Ground Floor Compass Building | Feldspar Close | Warrens Business Park |

Enderby | Leicester LE19 4SE | United Kingdom

Tel. +44 116 284 78 80 | Fax +44 116 284 78 81 | www.ivoclarvivadent.co.uk

Ivoclar Vivadent, Inc.

175 Pineview Drive | Amherst, N.Y. 14228 | USA

Tel. +1 800 533 6825 | Fax +1 716 691 2285 | www.ivoclarvivadent.us