Short Instructions

For
**Software
V4.0**
and higher

PrograMill

PM5
PM7

Milling machine

ivoclar

PrograMill PM7

PrograMill® PM5 + PM7

 en	Short Instructions Ceramic furnace	4
 de	Kurzanleitung Brennofen	22
 fr	Mode d'emploi simplifié Four de cuisson	41
 it	Istruzioni d'uso brevi Forno di cottura	60
 es	Instrucciones breves Horno de cocción	79
 sl	Kratka navodila Keramična peč	99
 hr	Kratke upute Peć za keramiku	117
 cz	Stručné pokyny Frézka	135
 sk	Stručný návod Keramická pec	153
 hu	Rövid utasítások Marógép	171
 sr	Кратко упутство Пећ за керамику	189
 mk	Кратки упатства Машина за режење	208
 bg	Кратки инструкции Пещ за керамика	227
 ro	Scurte instrucțiuni Mașina de frezat	246

1	About this Document	5
1.1	Signs and symbols	5
1.2	Supporting documents	7
2	Safety	8
2.1	Intended use	8
2.2	Operator's obligations	9
2.3	Personnel qualifications	9
2.4	Staff obligations	9
2.5	Personal protective equipment	10
2.6	Working areas, potential hazards and safety measures on the device	10
2.7	Safety instructions for potential hazard areas	11
2.8	Safety instructions for individual operating phases	13
3	Inserting Tools	15
4	Loading Materials	16
5	Filling the Tank	17
6	Starting a Milling Job	18
7	Cleaning	19

These Short Instructions form a part of the device documentation set. They are only valid in combination with the Operating Instructions of the device ("Supporting documents", page 7).

WARNING! Improper use of the device.

Risk to the patient and the user.

– These Short Instructions contain important information on the safety of the device: Please read the Operating Instructions and Short Instructions thoroughly prior to installation and operation.

These Short Instructions will offer you a quick introduction into the operation of PrograMill PM7 and PM5. Should you lose these Short Instructions, you can download them from www.ivoclar.com.

1.1 Signs and symbols

1.1.1 Warning and additional information

Warnings

Warnings are used in these Short Instructions to warn you of a risk of injury to persons and/or damage to property.

1. Please always read and observe these warnings.
2. Follow all measures marked with the warning symbol and warning word.

Depending on the severity and probability of the danger, the following warning levels are distinguished:

Warning symbol	Warning word	Danger level	Consequences resulting from non-observance
	DANGER	Immediate imminent danger	Death, severe personal injury
	WARNING	Potential danger	Death, severe personal injury
	CAUTION	Potential danger	Slight personal injury
–	NOTICE	Potential danger	Damage to property

Clarification of the structure of a warning:

WARNING! Name of source of danger, cause of hazard or type of risk.

Consequences of non-compliance with instructions.

– Action in order to avoid danger.

Hazard example:

WARNING! Risk of injury due to heavy lifting.

Injuries to the back due to overloading.

– Only lift the appliance with the aid of the fork lift.

– Unpack the device after transportation.

Additional informationen:

Symbol	Significance
	Additional information, e.g. for better understanding, for simplifying workflows or for further information

1.1.2 Signs and symbols in work instructions

Symbol	Significance
	Ensure the correct installation direction of the tools. Please observe: This symbol shows a PrograMill PM7 tool. Despite the difference in design, the procedure is identical for PrograMill PM7 and PrograMill PM5.
	Avoid inserting the tools in the wrong installation direction. Please observe: This symbol shows a PrograMill PM7 tool. Despite the difference in design, the procedure is identical for PrograMill PM7 and PrograMill PM5.
	Add new tools via the operating panel
	Check the status of the device or a device component, e.g. <ul style="list-style-type: none"> - Check whether the RFID label of a disc clamped into a disc holder is positioned centrally (applies to zirconium oxide, PMMA/DCL, wax) - Check the display, status or notification in the operating panel - Check the fill level of the tank
	Tap the key in the operating panel
	Ensure that the lock of the tank lid clicks into place properly
	Start a milling job via the operating panel
	Run daily cleaning tasks
	Run weekly cleaning tasks
	Run monthly cleaning tasks
	Document conducted cleaning intervals
	Follow the instructions of the operating panel
	The work instruction shown in the Short Instructions has been successfully carried out.

1.2 Supporting documents

For details on the applicable documents, please contact your local trade partner or go the download section of Ivoclar Vivadent AG at <http://www.ivoclar.com>

Document	Clarification
Operating Instructions PrograMill PM7	Comprehensive information on the operation of PrograMill PM7
Operating Instructions PrograMill PM5	Comprehensive information on the operation of PrograMill PM5
Checklist for your PrograMill System	Information about requirements that must be observed for transport, installation and use of the PrograMill system
Operating Instructions PrograMill Suction Unit	Information about the use of the PrograMill Suction Unit (optional)
Operating Instructions PrograMill Base	Information on the assembly and installation of the base cabinet (optional)
Operating Instructions PrograMill CAM	Handbook for the CAM Software "PrograMill CAM"
Safety Data Sheet PrograMill Fluid	Information about the use of the PrograMill Fluid coolant concentrate
Documentation on the discs / blocks used	Information on the use of discs / blocks

WARNING! Improper use of the device.

Risk to the patient and the user.

- The safety chapter contains important information on the safety of the device: Please read thoroughly prior to installation and operation.

The device is built according to state-of-the-art technology and recognized safety regulations. Nevertheless, its use could constitute a risk of injury of the user or third party or possibly cause impairment to the device and other property.

2.1 Intended use

PrograMill PM7 and PrograMill PM5 are designed for light to medium-duty milling and grinding in both wet and dry operation in the field of dental technology. Use the PrograMill PM7 and PrograMill PM5 exclusively for this purpose.

In the case of damage caused by improper use or failure to observe the Operating Instructions, all liability and guarantee claims are void.

The probe system that is available optionally as an accessory must only be used briefly for adjustment purposes and when the device is operated under supervision.

Please note that only the indications, materials and milling strategies approved by the manufacturer ensure processing without damage to the machine.

The machine has been designed for milling the following materials:

- Zirconium oxide (IPS e.max® ZirCAD)
- Glass-ceramics (IPS e.max® CAD, IPS Empress® CAD)
- PMMA materials (Telio® CAD, SR Vivodent® CAD, IvoBase® CAD)
- CoCr alloys (Colado® CAD)
- Titanium/titanium alloys (Colado® CAD)
- Waxes (ProArt CAD)

Only approved discs / blocks can be identified and thus used by the device via a corresponding RFID chip on the disc (RFID label or RFID container on the disc holder) or on the block holder (RFID container on the block holder), which ensure that misapplication is nearly impossible.

Basically, all Ivoclar materials and tools in their respective categories are suitable for processing.

All information regarding material and indication extensions corresponds to the state of knowledge at the time of going to print. For additional details, go to <http://www.ivoclar.com>.

It is not recommended to process materials or use tools from other manufacturers in this system due to the absence of milling strategy coordination and lack of tool harmonization.

2.1.1 Potential improper use

The device is not suitable for milling procedures outside of the dental field.

The following is considered as misuse of the system:

- Non-compliance with the specified intended use, intended operator specifications or intended environment
- Modifications, maintenance and repairs without prior authorization from Ivoclar
- Operation with inappropriate parameters

Improper use of the device can lead to the following:

- Danger for patient and operating personnel
- Impairment of the operability of the device

Ivoclar assumes no liability for damage resulting from improper use.

Typical misapplications of the device to be avoided:

Improper use	Consequences
Use of not approved materials or tools	Damage to the device
Incorrect cleaning (filter, milling chamber, changer)	<ul style="list-style-type: none"> - Skin irritations - Environmental damage - Damage to the device
Operation without coolant concentrate PrograMill Fluid	<ul style="list-style-type: none"> - Damage to the device - Service life of device and filter is reduced
Shut-down of the device by the user during the milling operation	<ul style="list-style-type: none"> - Damage to tools and discs or blocks - Calibration may be necessary
Operation without the tank (only applies in the case of wet operation)	Device will stop
Operation without the suction unit and filter	

2.1.2 Intended operator

The device may only be operated by qualified dental staff, dental technicians and CNC technicians who have received training for operating the device. The user is responsible for selecting the correct device settings.

2.1.3 Intended environment

The device is only permitted for use in closed rooms, within the specified ambient conditions (see Technical Data in the Operating Instructions, "Supporting documents", page 7) and in compliance with the requirements for safe installation (see Choosing the location site in the Operating Instructions, "Supporting documents", page 7).

2.2 Operator's obligations

The operator is responsible for the safe operation of the device.

- Ensure compliance and control:
 - Intended use
 - Statutory or other safety and accident prevention regulations
- Only operate the device in a technically perfect condition, in a proper, safety-conscious and risk-conscious manner and while observing the Operating Instructions ("Supporting documents", page 7).
- Keep these instructions and all applicable documents complete, legible and accessible to personnel at all times.

2.3 Personnel qualifications

- Ensure that the personnel assigned to work with the device have read and understood these instructions and all applicable documents, in particular safety, maintenance and repair information, before starting work.
- Ensure that personnel is aware of hazards and safety equipment ("Working areas, potential hazards and safety measures on the device", page 10).
- Manage the responsibilities, competence and monitoring of staff.
- All work must only be carried out by qualified technical staff.
- Personnel to be trained should only carry out work on the unit under the supervision of qualified technical personnel.

2.4 Staff obligations

- Only operate the device in a technically perfect condition, in a proper, safety-conscious and risk-conscious manner and while observing the Operating Instructions ("Supporting documents", page 7).
- Refrain from any process that could endanger staff or third parties.

- In the case of safety-related malfunctions, switch off the device immediately and allow the fault to be rectified by the authorized service partner.
- In all cases of doubt regarding the safety of the appliance, switch off the appliance and prevent further use.
- In addition to the overall documentation, legal or other safety and accident prevention regulations including applicable standards and guidelines of the respective operating country must be complied with.
- Do not wear jewellery such as rings, bracelets or watches when working on the appliance, especially when cleaning the milling chamber. Danger of crushing or cutting injuries.

2.5 Personal protective equipment

- If loud operating noise cannot be prevented (sound level ≥ 80 dBA), use hearing protectors during milling.
- For protection against milling dust, vapours or other particles of dental materials during cleaning work wear a protective mask (half mask with protection class FFP3).
- When carrying out cleaning work, wear gloves.
- When handling the PrograMill Fluid coolant concentrate and when cleaning the unit, wear suitable protective equipment (see Safety Data Sheet PrograMill Fluid, "Supporting documents", page 7).

2.6 Working areas, potential hazards and safety measures on the device

As an operator, you are working at the areas described below. The device is built according to state-of-the-art technology and recognized technical safety regulations. Hazardous areas, which cannot be avoided by design, are equipped with appropriate protective devices. Nevertheless, if the machine is used improperly, there is a danger to life or risk of injury to the user or third parties. In addition, the device, tool or workpiece may become damaged.

The working areas, their potential hazards and applicable safety measures are shown below:

No.	Working area	Tasks / Work	Safety measures	Protective effect
1 + 7	PrograMill Suction Unit (1) with separator (7), optional	See the Operating Instructions for the PrograMill Suction Unit ("Supporting documents", page 7)		
2	Service flap and drawer	Cleaning duties	Safety contact	When the flap is open, safety-relevant components will stand still.
3	Material changer	Placement of discs / blocks	Safety contact	- Protection against crushing - When the flap is open the device will stand still.
4	Milling chamber	Placement of tools. Cleaning when the device stands still.	Safety lock of the front flap. Access to the milling chamber is not possible during operation.	- Protection against crushing and cutting injuries from the tools -When the flap is open the device will stand still. - The flap cannot be opened when the device is running.
5	Connections (right side)	Connection to compressed air, water inlet/outlet, electricity, network connection	Cover, fuse, clearly identifiable connections	Protection against electric shock, health risks and danger to the environment due to leaking coolant fluid.
6	PrograMill Tank and Filter	Removal of the tank for filling with coolant fluid and filter exchange	Clearly marked tube connections	- Protection against health risks and danger to the environment due to leaking coolant fluid. - Without the tank, wet operation cannot be started.

2.7 Safety instructions for potential hazard areas

2.7.1 Electromagnetic waves

This device may cause interference to radio communications or interfere with the operation of equipment in the vicinity even if the applicable standards and guidelines are complied with.

- To avoid injury to the user, never remove the housing covers.
- Do not use devices that generate electromagnetic waves together in the same room as this device.
- Take appropriate measures in the case of radio interference, e.g. a new orientation, a new device position or isolating measures.
- Only use the original accessories recommended and supplied by the manufacturer. Failure to do so can result in increased interference emissions or decreased immunity of the device.

2.7.2 Mechanics

Whilst the device is in operation, there are the following risks in the milling chamber or in the material changer:

- Risk of crushing
- Cutting injuries caused by tools
- Danger of flying chips
- Damage to the machine
- To avoid injury to the user from inside the unit, never remove the covers, flaps or doors.

- Only operate the device when the flaps/ doors are shut. Never bypass safety equipment or put it out of operation, in particular the flaps/ doors.
- Do not reach into the material or tool changer.
- Do not open the flaps/ doors to the material or tool changer with force.
- Only reach into the milling chamber for the work described in the Operating Instructions and under the stated preconditions.

2.7.3 Sound emission

The sound emission strongly varies depending on the material being processed and the milling conditions (see Classifications in the Operating Instructions, "Supporting documents", page 7).

In the case of higher sound emissions, which may damage your hearing:

- Visually inspect the disc / the block and the tool.
- Check the fixation of the disc/ block in the disc or block holder.
- If loud operating noise cannot be prevented (sound level \geq 80 dBA), use hearing protectors during milling ("Personal protective equipment", page 10).

2.7.4 Electrical safety

If covers are opened or parts are removed which are only accessible with tools, voltage-carrying components may be exposed. The plugs may also be under voltage. There is a risk of electric shock!

- Electrical work must only be carried out by your authorized service partner.
- Before connecting the device, check that the supply voltage and frequency are correct at the point of installation (see Electrical data and information on the type plate in the Operating Instructions, "Supporting documents", page 7).
- To avoid injury to the user and for cooling reasons and fire protection, never remove the housing covers.
- Observe the manufacturer's installation instructions for house installation ("Supporting documents", page 7, Checklist for your PrograMill system).
- To avoid the risk of electric shock, only connect the device to the power supply with a protective conductor, do not reach into the connections on the right side of the housing or manipulate them and do not reach into the housing interior via the service flap, the material changer or tool changer.
- The mains plug is used as a supply circuit disconnecting means. Always connect the mains plug to an easily accessible protective contact socket.
- Protect the cable against damage due to heat, moisture, oil, sharp edges, pressure and crushing. In the case of damaged cables, replace only with the original spare parts.
- The device and the suction unit must be connected to a grounded and freely accessible protective contact socket, each in a separately protected circuit. Make sure that there are no devices connected which cause strong mains voltage fluctuations when they are switched on. These fluctuations interfere with the electronic controls and may cause failure of the system.
- Make sure that the protective contact sockets are equipped with a residual current circuit breaker (FI).
- If you need to disconnect the unit from the power supply at a later time: Disconnect the plug from the mains socket, not on the device.

2.7.5 Pneumatics

The pneumatic components in this device are under high pressure. Sudden escape of compressed air through leaking pipes and connections or swirling hoses can cause serious injuries and damage to hearing.

- Ensure that only personnel with additional pneumatic training perform work on pneumatic components.
- Ensure that the compressed air supply meets the required system specification (see Requirements on incoming compressed air in the Operating Instructions, "Supporting documents", page 7). Do not exceed the maximum permissible pressure.
- Ensure that the compressed air is clean, oil-free and dry.
- Ensure that the pneumatic components are checked regularly.
- Ensure that all defects, such as loose connections, damaged cables, etc., are dealt with immediately.

Before working on the pneumatics:

- Check the main shut-off valve.
- Switch the device off (see Operating Instructions, "Supporting documents", page 7).
- Before opening or removing a tube, make sure that it is not pressurized.

2.7.6 Hazardous materials

Milling materials

Depending on the material of the milled discs / blocks, without appropriate safety measures during processing there is a risk respiratory problems caused by inhaling hazardous milling dust or chips, residues or emissions.

- Only use the device when the suction is switched on.
- Use only discs / blocks that do not present a hazard to health when the suction system is properly used.
- Only operate the suction system with the filter specified by the manufacturer.
- In order to prevent subsequent work from being damaged, always clean the device before and after processing CoCr alloys.
- When extracting carcinogenic materials and/or welding fumes from materials with a nickel or chrome content, the technical rules for the handling of hazardous substances TRGS 560 regarding air recirculation in connection with the treatment of carcinogenic hazardous materials must be complied with.

Coolant fluid/concentrate

- Without appropriate safety measures there is a risk of skin irritations, allergic reactions and respiratory problems.
- Observe the specifications in the Safety Data Sheet PrograMill Fluid ("Supporting documents", page 7).

2.8 Safety instructions for individual operating phases

2.8.1 Transport

- Only transport the device according to the description in the Operating Instructions (see Transportation and Assembly in the Operating Instructions, "Supporting documents", page 7).
- In order to avoid corrosion on the device and subsequent damage to the device, transport and store the device only within the permissible temperature range and ambient conditions (see Requirements on ambient conditions in the Operating Instructions, "Supporting documents", page 7).

2.8.2 Initial operation

- Ensure that this machine is only operated by trained specialists.
- If the device has been stored in a cold environment or at high humidity, maintain a drying or temperature adjustment time of approx. 4 hours (without voltage) before initial operation, at room temperature.
- Before connecting the device, check that the supply voltage and frequency are correct at the point of installation (see Electrical Data in the Operating Instructions, "Supporting documents", page 7).
- Before connecting the device, check that the compressed air connections are correct at the point of installation (see Requirements on incoming compressed air, "Supporting documents", page 7).
- Comply with the requirements of ambient conditions (see Requirements on location site and ambient conditions in the Operating Instructions, "Supporting documents", page 7).

2.8.3 Operation

- Keep unauthorized persons, such as patients, children and animals, away from the unit.
- In all cases of doubt regarding the safety of the appliance, switch off the appliance and take suitable measures to prevent further use.
- Prior to connecting power or operation, check the device, the accessories and protective equipment for any damage.
- Do not use damaged, non-functioning equipment or accessories; instead notify your authorized service partner.
- In order to ensure the product safety and warranty services, the device must be exclusively operated with the original accessories from Ivoclar. The user bears the risk when using non-approved accessories.
- Ivoclar recommends using approved materials only to ensure process reliability.
- Note on the processing of titanium/titanium alloys: Process these material only in combination with coolant fluid. Risk of ignition of the chips!
- Never bypass the safety equipment of the machine or set it out of operation ("Working areas, potential hazards and safety measures on the device", page 10).
- Only operate the device when the milling chamber, material changer and service flap are closed and when the suction unit is connected and the filled tank/filter in place.

- To avoid damaging the unit and reducing the performance, follow the cleaning instructions and cycles ("Cleaning", page 19 and Cleaning guidelines in the Operating Instructions, "Supporting documents", page 7).
- Only operate the device unsupervised if the operating conditions for unsupervised operation described below are fulfilled.

Unsupervised operation:

The machine may be operated unsupervised, provided the national and local laws and provisions allow for such action and provided that they are observed. Furthermore, the requirements of the respective insurance company must be met.

The following conditions must be complied with in order to guarantee unattended operation.

- Never use the device if the milling chamber is heavily soiled.
- Protect the device against unauthorized access.
- Ensure that the room in which the device will operate is equipped with an automatic fire alarm system.

Patients with cardiac pacemakers or pacemaker electrodes:

- During operation, there may be interference with other electrical medical equipment.
- Patients with cardiac pacemakers or pacemaker electrodes are at risk from the following potential hazards:
 - Interference with the pacemaker function
 - Damage to the cardiac pacemaker
- If in doubt, either you or your patient should get in touch with a cardiologist.

2.8.4 Cleaning by the user

- Only clean as specified in these Short Instructions and the Operating Instructions and observe the associated safety regulations ("Cleaning", page 19 and Cleaning guidelines in the Operating Instructions, "Supporting documents", page 7).

2.8.5 Improper maintenance, modifications and repairs

Improper service and improper repairs or modifications will endanger patients and users and result in damage to the unit.

Should you carry out any repairs or maintenance to the system or modifications to the device or remove the housing without prior written consent by an authorized service partner, all warranty claims are void.

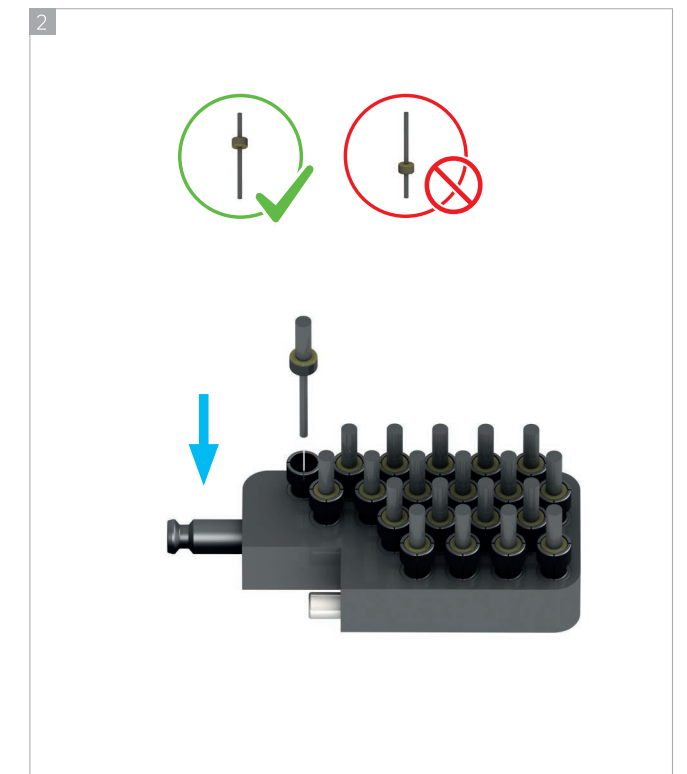
Unauthorized opening and removal of components can expose voltage-carrying components. The plugs may also be under voltage. There is a risk of electric shock!

- Maintenance as well as repairs of damaged safety equipment or machine parts must only be carried out by a service partner authorized by Ivoclar.


3 Inserting Tools

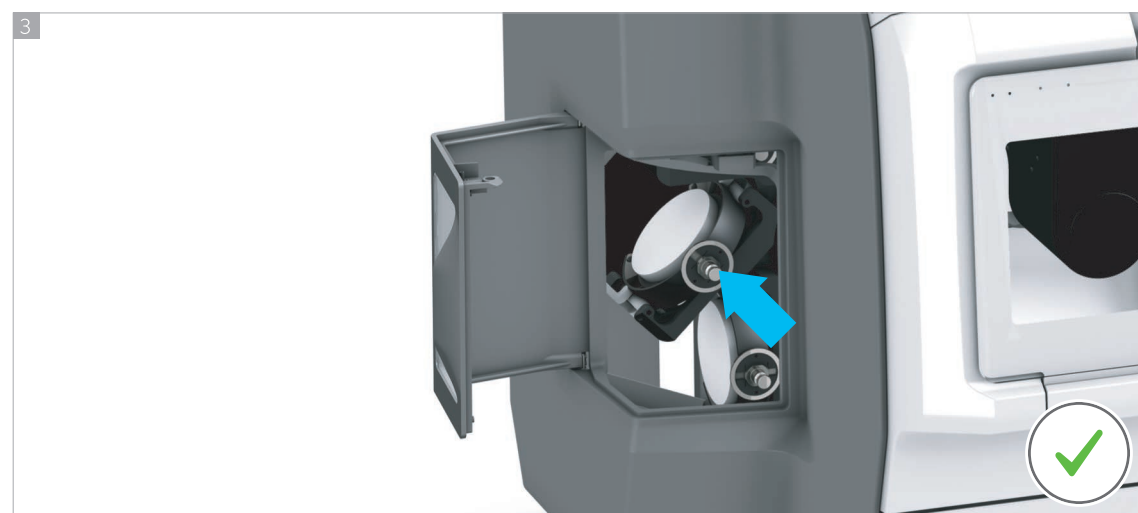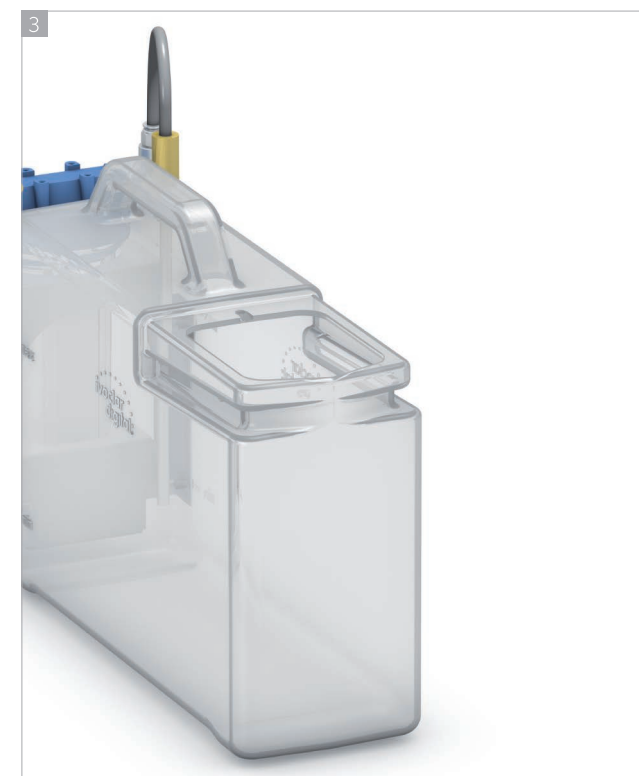
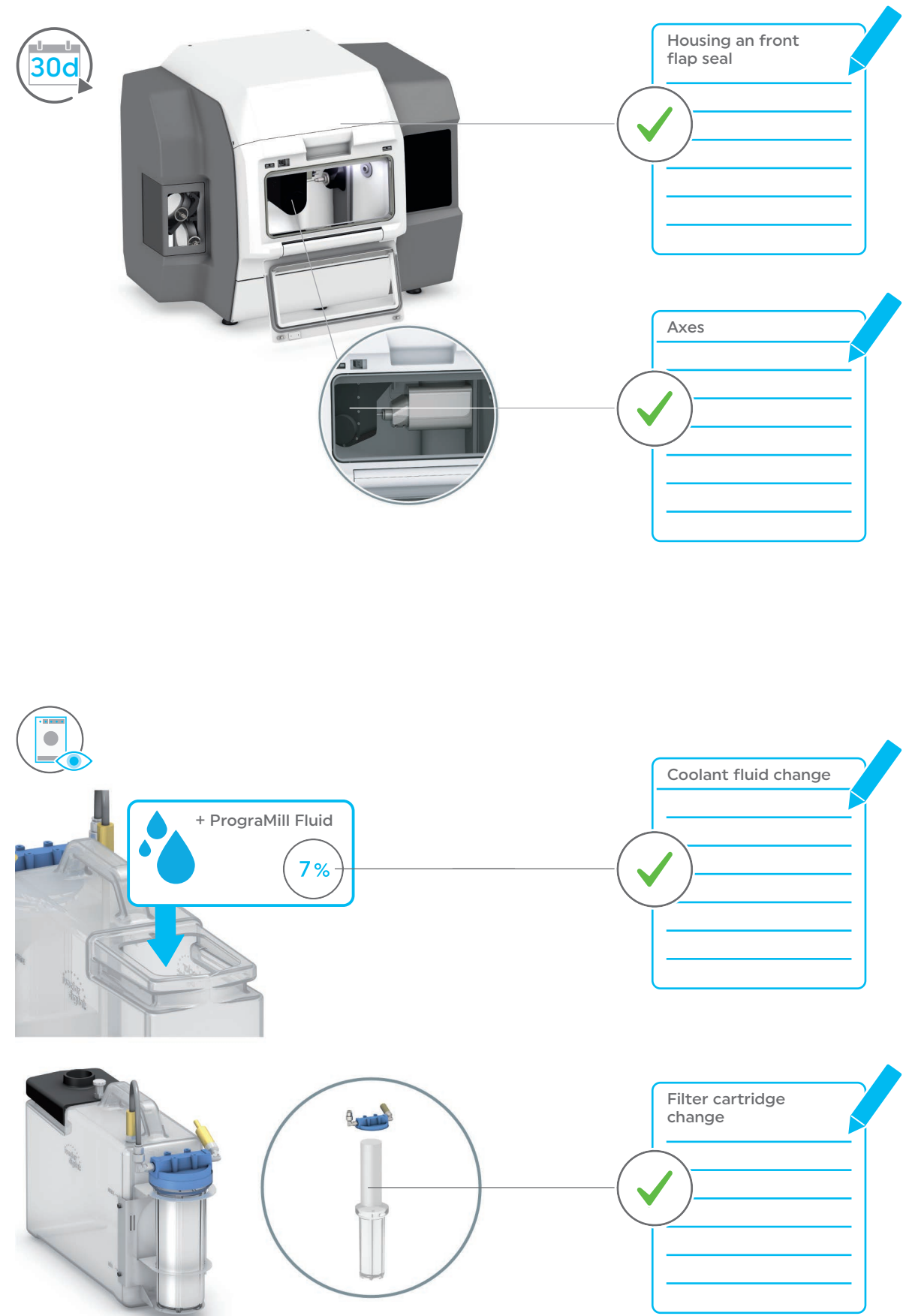
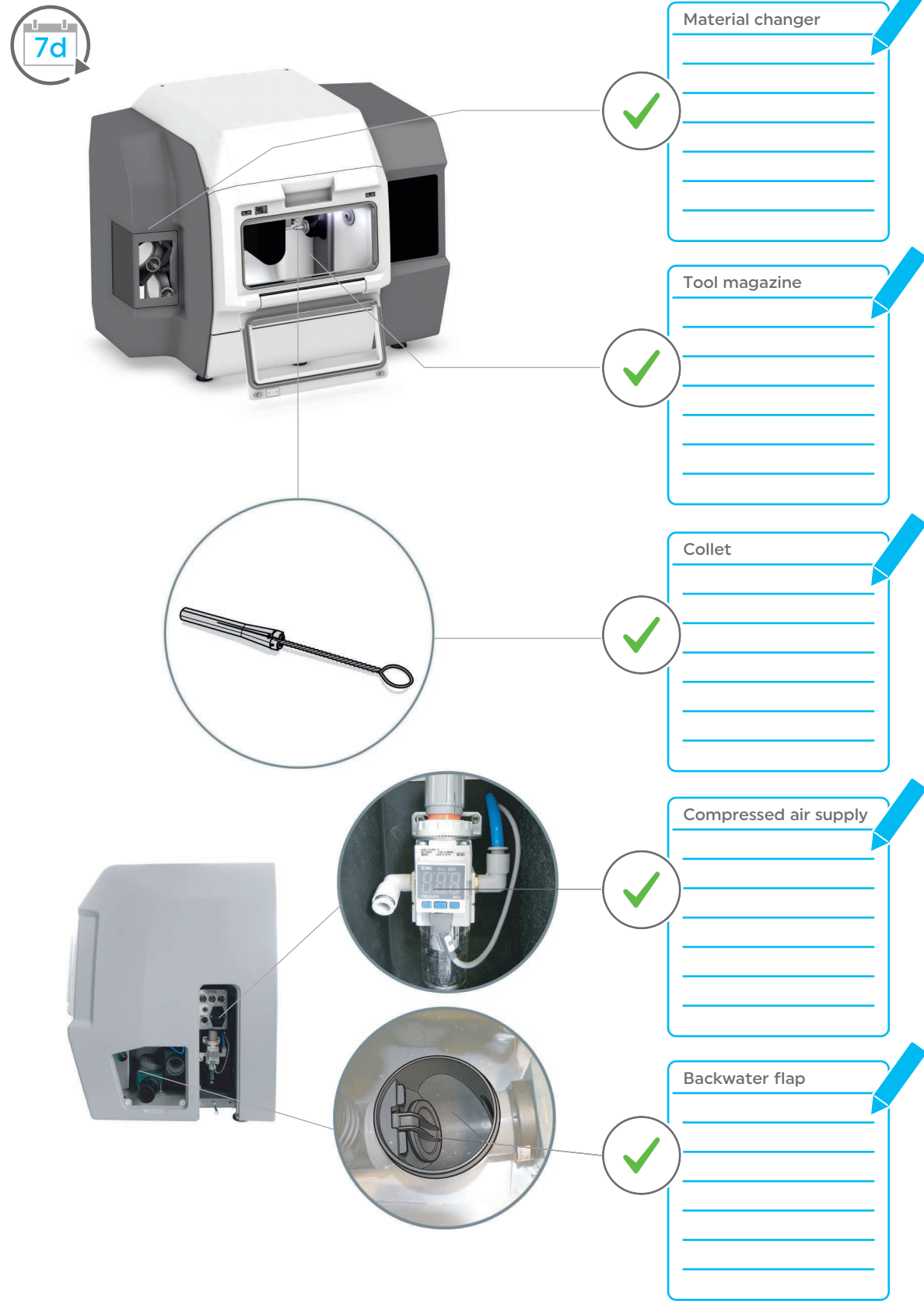
Please observe: The illustrations in the following sections show PrograMill PM7 with the corresponding tool magazine and tools. Despite the difference in design, the procedure is identical for PrograMill PM7 and PrograMill PM5.

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© Ivoclar Vivadent AG, Schaan/Liechtenstein