PrograPrint PR5®

Short Instructions
Kurzanleitung
Guide d'initiation rapide
Istruzioni d'uso brevi
Guía breve
Instruções de Uso Resumidas

Printing

Making People Smile

ivoclar

PrograPrint PR5[®]

Table of Contents | Inhaltsverzeichnis | Table de matières | Indice | Índice | Lista de Conteúdo

 en	Short Instructions	4
 de	Kurzanleitung	14
 fr	Guide d'initiation rapide	24
 it	Istruzioni d'uso brevi	34
 es	Guía breve	44
 pt	Instruções de Uso Resumidas	54
	Instructions Anleitungen Instructions Istruzioni Instrucciones Instruções	64

English

1. About this document

These Short Instructions are part of the overall documentation for the device. They are only valid in combination with the Operating Instructions of the device ("Documents of equal applicability and required files", Page 5).

WARNING! Improper use of the device.

Risk to the patient and the user.

► The Short Instructions contain important information on the safe handling of the device: Operating Instructions and Short Instructions must be read thoroughly prior to installation and operation.

These Short Instructions will help you to quickly familiarize yourself with the use of PrograPrint PR5.

Should you lose these Short Instructions, you can download them from www.ivoclar.com.

1.1. Signs and symbols

1.1.1 Warnings and additional information

Warnings

Warnings are used in these Short Instructions to warn you of a risk of injury to persons and/or damage to property.

- Please always read and observe these warnings.
- Follow all measures marked with the warning symbol and warning word.

Depending on the severity and the probability of the danger, the following warning levels are distinguished.

Warning symbol	Warning word	Danger level	Consequences resulting from non-observance
	DANGER	Immediate imminent danger	Death, severe personal injury
	WARNING	Potential danger	Death, severe personal injury
	CAUTION	Potential danger	Slight personal injury
	NOTICE	Potential danger	Damage to property

Explanation of the structure of a warning:

WARNING WORD! Name of source of danger, cause of hazard or type of risk.

Consequences of non-compliance with instructions.

► Action in order to avoid danger.

Example of a warning:

WARNING! Risk of injury due to heavy lifting.

Injuries to the back due to overloading.

- Only lift the device with the aid of a pallet truck.
- Unpack the device only after transportation.

Additional information

Symbol	Meaning
	Additional information, e.g. for better understanding, for simplifying workflows or for further information.

1.2. Documents of equal applicability and required files

For details on the documents of equal applicability, please contact your local trade partner or go to the download section of Ivoclar at www.ivoclar.com.

In the download section, you can find additional files, such as:

- DME file for the software "Dental System"

Document	Explanation
Operating Instructions PrograPrint PR5	Comprehensive information on the use of PrograPrint PR5
Instructions for Use for the different materials	Information about the use of the materials to be processed
Operating Instructions for the cleaning unit PrograPrint Clean	Information about requirements that must be observed for the transport, set-up and use of PrograPrint Clean
Operating Instructions for the post-curing unit PrograPrint Cure	Information about requirements that must be observed for the transport, set-up and use of PrograPrint Cure
Instructions for Use for PrograPrint CAM	Handbook for the PrograPrint CAM software
Short Instructions for: <ul style="list-style-type: none"> - PrograPrint PR5 (this document) - PrograPrint Clean - PrograPrint Cure 	Excerpt of operation-relevant topics from the respective Operating Instructions

2. Safety

WARNING! Improper use of the device.

Hazard to the user.

► The safety chapter contains important information on the safety of the device: Please read it thoroughly prior to installation and operation.

The device is built according to state-of-the-art technology and recognized safety regulations. Nevertheless, operation bears the risk of injury to the user or third parties. Moreover, impairments of the device and other property are possible.

2.1. Intended use

PrograPrint PR5 has been designed for 3D printing in dental technology.

Use the PrograPrint PR5 exclusively for this purpose.

In the case of damage caused by improper use or failure to observe the Operating Instructions, all liability and guarantee claims are void.

Please note that only the materials and cleaning agents approved by the manufacturer ensure processing without damage to the machine.

All information regarding material and indication extensions corresponds to the state of knowledge at the time of going to print.

For additional details, go to <http://www.ivoclar.com>.

The Ivoclar Download Center is available at <https://downloadcenter.ivoclar.com/#lang=en>

The use of non-approved materials from other manufacturers is not recommended due to the lack of coordination with the printing/fabrication process.

2.1.1 Potential improper use

The device is not suitable for 3D printing procedures outside of the dental field.

The following is considered as misuse of the system:

- Non-compliance with the specified intended use, intended operator specifications or intended environment
- Modifications, maintenance and repairs without prior authorization from Ivoclar
- Operation with inappropriate parameters

Improper use of the device can lead to the following:

- Danger for patients and operating personnel
- Impairment of the operability of the device

Ivoclar assumes no liability for damages resulting from improper use.

Typical misapplications of the device to be avoided:

Improper use	Consequences
Use of non-approved materials	The device cannot execute any printing jobs
Incorrect cleaning and disposal (build platform, build chamber, hood etc.)	<ul style="list-style-type: none">- Skin irritations- Environmental damage- Damage to the device
Operation without cartridge, tank or build platform	Damage to the device
Operation with incorrectly inserted tank or build platform	<ul style="list-style-type: none">- Damage to the device- Deficient printed object
Shut-down of the device by the user during printing	Deficient printed object

Improper use	Consequences
Mixing of different materials	<ul style="list-style-type: none">- Damage to the patient's health- Cross-contamination of materials- Deficient printed object- Damage to the material, cartridge and material bottle
Failure to observe the specified fill level when filling the tank manually	<ul style="list-style-type: none">- In case of overfilling: Unpolymerized material will overflow during printing- In case of insufficient material filling: Material will run out during printing- Deficient printed object

2.1.2 Intended user

The device may only be operated by dental technicians and qualified dental staff for the fabrication of dental objects for dental applications.

The user is responsible for selecting the correct device settings.

2.1.3 Intended environment

The device is only permitted for use in closed rooms within the specified ambient conditions (see technical data in the Operating Instructions, "Documents of equal applicability and required files", Page 5) and in compliance with the requirements for safe installation (see requirements regarding the location site, "Documents of equal applicability and required files", Page 5).

2.2. Operator's obligations

The operator is responsible for the safe operation of the device.

- ▶ Ensure compliance and control:
 - Intended use
 - Statutory or other safety and accident prevention regulations

- ▶ Only operate the device in a technically perfect condition, in a proper, safety-conscious and risk-conscious manner and while observing the Operating Instructions ("Documents of equal applicability and required files", Page 5).
- ▶ Keep these instructions and all documents of equal applicability complete, legible and accessible to personnel at all times.

2.3. Personnel qualifications

- ▶ Ensure that the personnel assigned to work with the device have read and understood these instructions and all documents of equal applicability, in particular safety, maintenance and repair information, before starting work.
- ▶ Ensure that personnel is aware of hazards and safety equipment ("Working areas, potential hazards and safety measures on the device", Page 8).
- ▶ Manage the responsibilities, competence and monitoring of staff.
- ▶ All work must only be carried out by qualified technical staff.
- ▶ Personnel to be trained should only be working with the device under the supervision of qualified technical staff.

2.4. Staff obligations

- ▶ Only operate the device in a technically perfect condition, in a proper, safety-conscious and risk-conscious manner and while observing the Operating Instructions ("Documents of equal applicability and required files", Page 5).
- ▶ Refrain from any process that could endanger personnel or third parties.
- ▶ In the case of safety-related malfunctions, switch off the device immediately and allow the fault to be rectified by the person responsible.
- ▶ In all cases of doubt regarding the safety of the device, switch off the device and prevent further use.
- ▶ In addition to the overall documentation, legal or other safety and accident prevention regulations including the applicable standards and guidelines of the country in which the device is operated must be complied with.
- ▶ Do not wear jewellery such as rings, bracelets or watches when working on the device, especially when cleaning the build chamber, to prevent cutting injuries or bruising.

2.5. Personal protective equipment


- ▶ If loud operating noises cannot be prevented, use hearing protectors during printing.
- ▶ For the protection against material and cleaning agent vapours or other dental material particles while working with cartridges or material bottles or during cleaning, wear protective masks (half mask with protection class FFP3), gloves and protective gear suitable for the respective material (see Instructions for Use of the materials being used, "Documents of equal applicability and required files", Page 5).

2.6. Working areas, potential hazards and safety measures on the device

As an operator, you are working at the areas described below.

The device is built according to state-of-the-art technology and recognized technical safety regulations. Hazardous areas, which cannot be avoided by design, are equipped with appropriate protective devices. Nevertheless, if the machine is used improperly, there is a danger to life or risk of injury to the user or third parties. In addition, the devices or dental applications may become damaged.

The working areas, their potential hazards and applicable safety measures are shown below:

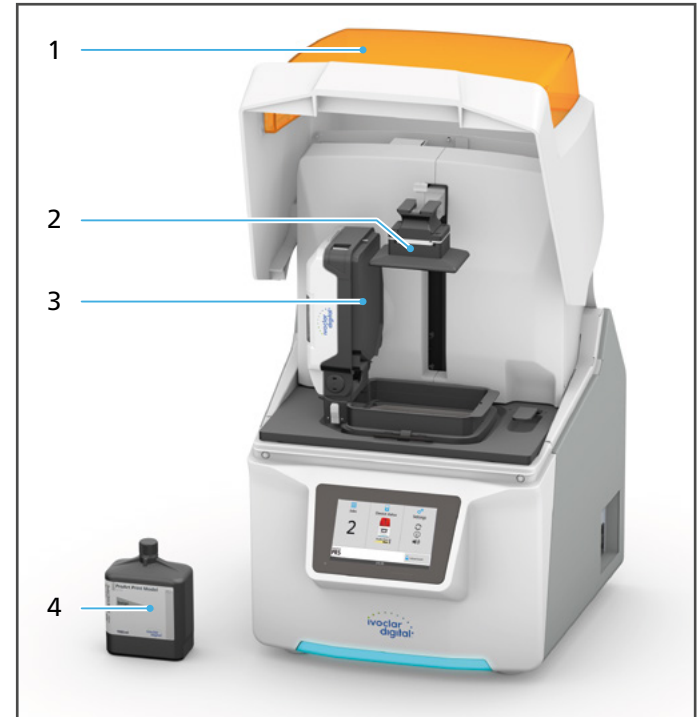The hood (1) of the printer can only be opened once the build platform (2) is in the home position (highest position) after a print job, as depicted. As soon as the movement of the build platform is complete, the hood is automatically unlocked.

No.	Working area	Tasks / Work	Type of hazard	Safety equipment	Protective effect
1	Hood	<ul style="list-style-type: none"> - Access to the build chamber - Mounting the build platform and cartridge - Cleaning of the build chamber with the printer switched off 	Mechanical hazards	<ul style="list-style-type: none"> - Safety lock of the hood. Access to the build chamber is not possible while the build platform is moving. - The printer does not start with the hood open. 	Protection against crushing
			<p>Premature polymerization of the material in the cartridge due to incident light.</p> <p>Contamination of the material.</p>	<ul style="list-style-type: none"> - Cushioning of the hood against slamming shut - Acoustic signal (Close hood) - Tinted window 	<ul style="list-style-type: none"> - Protection against damage to the printer - Protection of the material
2	Build platform	Creation of the printed objects	Mechanical hazards by the build platform falling down	<ul style="list-style-type: none"> - Construction-safe design - Fixation lever 	Protection against crushing
			Leaking, slopping over material due to the build platform falling down into the filled cartridge		<ul style="list-style-type: none"> - Protection against health impairments - Protection against environmental damage
3	Cartridge	Material supply for print jobs	Leaking, slopping over material during transport of the cartridge.	<ul style="list-style-type: none"> - Handle at the bottom section, not the top section - Accidental swivelling impeded by the handle at the bottom section 	<ul style="list-style-type: none"> - Protection against health impairments - Protection against environmental damage
			Dripping build platform	Wrap around rim at the top edge	
4	Material bottle	Material supply for print jobs	Leaking, slopping over material during transport, opening or handling of the bottle	<ul style="list-style-type: none"> - Robust design of the material bottle - Material bottle can be held stable and does not give in to pressure 	<ul style="list-style-type: none"> - Protection against health impairments - Protection against environmental damage

2.7 Safety instructions for potential hazard areas

2.7.1 Electromagnetic waves

Even though the valid standards are observed, it is possible that the device reacts to radio frequency interference or the operation of other devices in the immediate vicinity.

- ▶ Do not operate devices that generate electromagnetic waves in the same room as this device.
- ▶ Take appropriate measures in the case of radio interference, e.g. a new orientation, a new device position or electromagnetic shielding.
- ▶ Only use the original accessories recommended and supplied by the manufacturer. Failure to do so can result in increased interference emissions or decreased immunity of the device.

2.7.2 Mechanics

Whilst the device is in operation, there are the following risks in the build chamber:

- Risk of crushing
- Risk of damage to the device
- ▶ To prevent injury of the user in the inner chamber of the device, never remove the housing cover and do not open the hood by force while the device is in operation.
- ▶ Do not use the device when the hood is damaged (cracks, holes, missing window etc.).

2.7.3 Electrical safety

If covers are opened or parts are removed which are only accessible with tools, voltage-carrying components may be exposed. The plugs may also be under voltage. There is a risk of electric shock.

- ▶ Electrical work must only be carried out by your authorized service partner.
- ▶ Before connecting the device, check that the supply voltage and frequency are correct at the point of installation (see electrical data and the information on the type plate in the Operating Instructions, "Documents of equal applicability and required files", Page 5).
- ▶ To avoid injury to the user and for cooling reasons and fire protection, never remove the housing covers.
- ▶ For installation, observe the requirements of the manufacturer for house installation.

- ▶ To prevent the risk of electric shock, connect the device only to a power supply with a protective conductor.
- ▶ The mains plug is used as disconnect device. Always connect the mains plug to an easily accessible protective contact socket.
- ▶ Connect the grounded and freely accessible protective contact socket to a separately secured circuit.
- ▶ Make sure that the protective contact socket is equipped with a residual current device (RCD).
- ▶ Connect the printer to a separately secured circuit or make sure that no devices are connected that cause severe supply voltage fluctuations when switched on. These fluctuations interfere with the electronic controls and may cause failure of the system.
- ▶ Disconnect power before cleaning and maintenance work approved to be performed by the user.
- ▶ If you need to disconnect the unit from the power supply at a later time: Disconnect the plug from the mains socket, not on the device.

2.7.4 Hazardous materials

Incorrect handling of the device, of the accessories or non-compliance with the processes described in the Operating Instructions may lead to contact of the user with unpolymerized material. This may result in health damages, e.g. skin irritations. Depending on the material, there is a risk of skin irritations, allergic reactions or respiratory diseases without the corresponding safety precautions.

- ▶ Observe the Instructions for Use of the materials used ("Documents of equal applicability and required files", Page 5).
- ▶ Wear protective gear ("Personal protective equipment", Page 7).
- ▶ Make sure that the room is sufficiently ventilated when working with the materials or handling the material bottles.

2.8. Safety information for the individual operating phases

2.8.1 Transport

- ▶ Transport the device only as described in the Operating Instructions (see requirements regarding transport and setting up in the Operating Instructions, "Documents of equal applicability and required files", Page 5).
- ▶ In order to avoid corrosion on the device and subsequent damage to the device, transport and store device only in the permissible temperature range and ambient conditions (see requirements regarding the ambient conditions in the Operating Instructions, "Documents of equal applicability and required files", Page 5).

2.8.2 Initial operation

- ▶ Ensure that this device is only operated by authorized and trained specialists.
- ▶ If the device has been stored in a cold environment or at high humidity, maintain a drying or temperature adjustment time of approx. 4 hours (without voltage) at room temperature before initial operation.
- ▶ Before connecting the device, check that the supply voltage and frequency are correct at the point of installation (see electrical data in the Operating Instructions, "Documents of equal applicability and required files", Page 5).
- ▶ The requirements regarding the location site and ambient conditions must be observed (see requirements regarding the location site and ambient conditions in the Operating Instructions, "Documents of equal applicability and required files", Page 5).

2.8.3 Operation

- ▶ Ensure that this device is only operated by authorized and trained specialists.
- ▶ Keep unauthorized persons, such as patients and children, as well as animals away from the unit.
- ▶ In all cases of doubt regarding the safety of the device, switch off and disconnect the device and take suitable measures to prevent further use.
- ▶ Prior to connecting power or operation, check the device, the accessories and protective equipment for any damage.
- ▶ Do not use damaged, non-functioning equipment or accessories; instead notify your authorized service partner.
- ▶ In order to ensure the product safety and warranty services, the device must be exclusively operated with the original accessories from Ivoclar, particularly the original power cord. The user bears the risk when using non-approved accessories.
- ▶ Ivoclar recommends using only approved materials to ensure process reliability.
- ▶ Never bypass the safety equipment of the device or set it out of operation ("Working areas, potential hazards and safety measures on the device", Page 8).
- ▶ Only operate the device when the build chamber is closed.
- ▶ To prevent damage to the device and a reduction of the device performance, observe the cleaning requirements and cycles (see "Cleaning by the user", Page 12 in these Short Instructions as well as the overview and instructions for the cleaning and maintenance work in the Operating Instructions, "Documents of equal applicability and required files", Page 5).
- ▶ Only operate the device unsupervised if the operating conditions for unsupervised operation described below are fulfilled.

Unsupervised operation:

The device may be operated unsupervised, provided the national and local laws and provisions allow for such action and provided that they are observed. Furthermore, the requirements of the respective insurance company must be met.

- ▶ Never use the device if the build chamber is heavily soiled.
- ▶ Protect the device against unauthorized access.

2.8.4 Cleaning by the user

- ▶ Only clean as specified in the Operating Instructions and observe the related safety regulations (see overview and requirements regarding cleaning and maintenance work in the Operating Instructions, "Documents of equal applicability and required files", Page 5).

2.8.5 Improper maintenance, modifications and repairs

Improper service and improper repairs or modifications will endanger patients and users and result in damage to the device and the final products.

Should you carry out any repairs or maintenance of the device or make modifications or remove the housing without prior written consent by an authorized service partner, all warranty claims are void.

Unauthorized opening and removal of components can expose voltage-carrying components. The plugs may also be under voltage. There is a risk of electric shock!

- ▶ Maintenance as well as repairs of damaged safety equipment or machine parts must only be carried out by a service partner authorized by Ivoclar.

3. Additional information

- Download link Operating Instructions PrograPrint System:
<https://www.ivoclar.com/prograprint-manual>
- Download link software and Test Files:
https://www.ivoclar.com/en_li/tools/software-updates

Ivoclar Vivadent AG herewith confirms that the radio equipment of PrograPrint PR5 (dental laboratory device) complies with the directive 2014/53/EEC. The entire text of the EU Declaration of Conformity can be found under the following link:
<https://www.ivoclar.com/prograprint-manual>

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Ivoclar Vivadent Inc.