



Variolink® Veneer / Syntac®

Light-cure Resin Cement

Placement of ceramic restorations where adequate light penetration is available


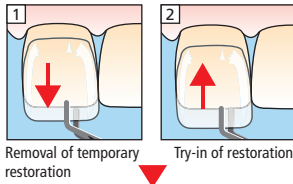


Restoration

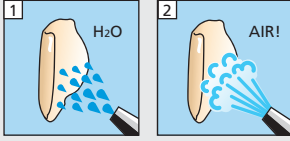


Pre-treatment of restorations according to manufacturer's instructions


Monobond Plus

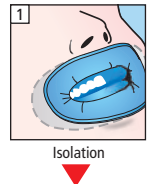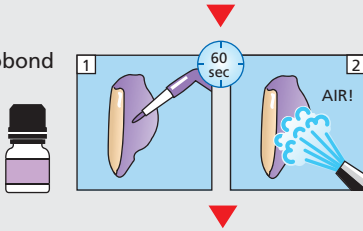


Preparation

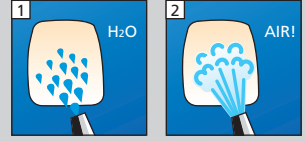





Enamel Total Etch

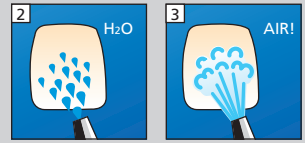


Syntac Primer

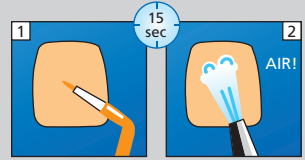

Syntac Adhesive

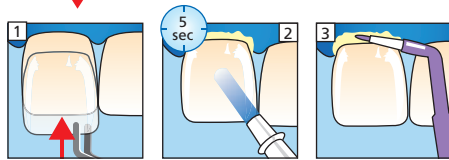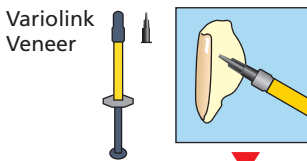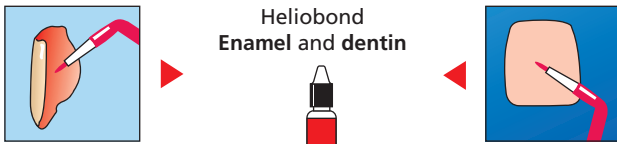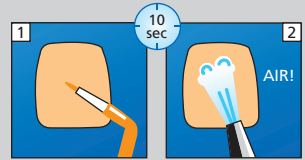


Liquid Strip

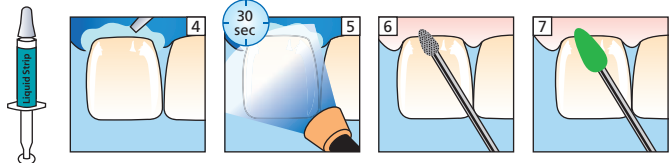


Variolink® Veneer / Syntac®

Lichthärtendes Befestigungscomposito

Für die rein lichthärtende Befestigung von transluzenten Vollkeramiken


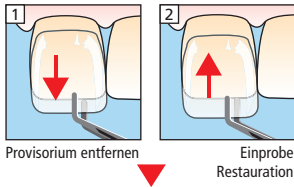


Restauration

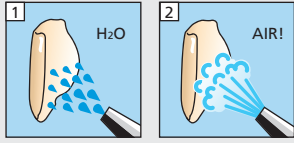


Vorbereitung der Restauration nach Herstellerangaben


Monobond Plus

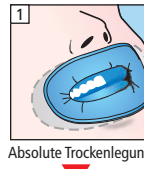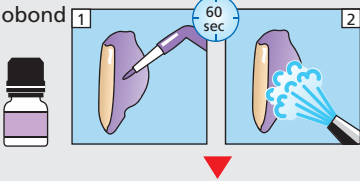


Präparation

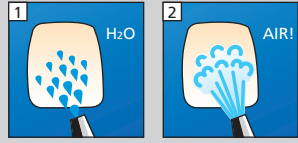



Schmelz Total Etch

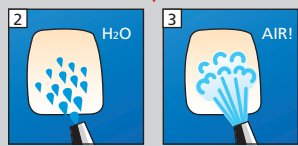


Syntac Primer

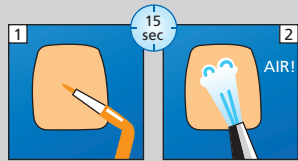



Syntac Adhesive

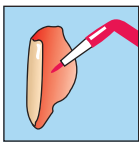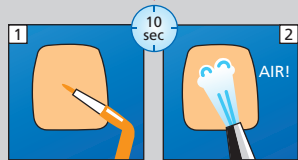

Heliobond Schmelz und Dentin

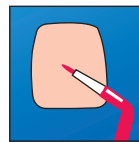

Variolink Veneer

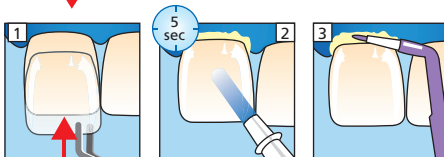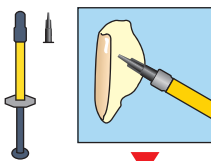

Liquid Strip

